

AnchorFast SlimFit Oral Endotracheal Tube Fastener

Introduction

This is a skills checklist for the application, routine care and removal of the AnchorFast SlimFit oral endotracheal tube fastener. The oral endotracheal tube fastener is indicated for use by healthcare professionals in securing oral endotracheal tubes ranging in size from 5.0 to 8.0 mm inner diameter. The suitability of the oral endotracheal tube fastener must be assessed for each patient.

Date Completed	Name	Employee Number	Dept.
----------------	------	-----------------	-------

Criteria (Refer to Instructions for Use for detailed information)	Eval Method*	Completed
Prepare the Skin		
1. Skin should be clean, dry, and free of oily residue		
2. Do not use skin gel wipes or other brands of skin preps with this device		
Apply the Oral Endotracheal Tube Fastener		
1. Remove the product from packaging		
2. Separate the hook and loop closure (gray tab) of the neck strap		
3. Remove the release liners from the two skin barriers		
4. Pull back gently on the skin barrier pads so they are out of the way, to prepare for application		
5. Center the device on the patient's upper lip, so the non-absorbent upper lip foam lightly touches the skin		
6. Press the two skin barrier pads on the patient's skin and hold in place until they adhere well. This should take approximately 30 seconds.		
Secure the ET Tube		
1. Squeeze the tabs on the sides of the gliding tube shuttle and move the clamp along the track to a location adjacent to the ET Tube		
2. Before applying the wrap to the ET tube, make sure the tube is dry and free of any residue. Remove the release liner from the "ET tube wrap" [wrap] exposing the adhesive.		
3. Position the ET tube under the non-slip grippers. Loop the wrap tightly around the ET tube; avoid including any other tubes within the wrap. Pull the remaining portion of the wrap through the security clamp.		
4. Secure the wrap by snapping shut the one-click security clamp (an audible click will be heard)		
Apply the Adjustable Neck Strap		
1. Adjust the neck strap at the gray tab		
2. Do not overtighten. Allow two fingers width between the strap and the back of the patient's head.		

(Continued)

INSERVICING SCRIPT & SKILLS CHECKLIST

Routine Care		
1. Reposition the tube by squeezing the gliding tube shuttle tabs on the outer edges and move in either direction along the track. Reposition the ET tube from side-to-side at least every two hours or more frequently if the patient's condition dictates, to minimize the risk of injury to the skin and/or lips from unrelieved pressure and shear forces.		
2. To minimize the risk of pressure injury, inspect the patient's lips and skin at least every two hours or more frequently if the patient's condition dictates (e.g. vulnerable to fluid shifts and/or edema)		
3. Be sure to frequently assess patient since wear time varies by patient		
4. Reconfirm position, depth of intubation, and patency of the ET tube or other airway device during and after any change in the patient's head, neck, or body position, or any change in the location of the fixation device		
5. Use caution during patient movement and/or repositioning to avoid dislodging the ET tube		
Removal		
1. Release the security clamp holding the wrap in place		
2. Carefully remove the wrap from around the ET tube		
3. Release the neck strap by unfastening the hook and loop closure at the gray tab		
4. Remove the skin barrier pads by gently peeling them away from the patient's skin		

* Evaluation Method: (L) return demos live in class or skill lab, (O) observation in worksetting, (CR) chart review, (V) verbalizes information, (R) read only – not performed – for information, (NA) not applicable.

Reviewed by (signature) _____.

USA: Rx Only

* **CAUTION:** FEDERAL (USA) LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN OR OTHER HEALTHCARE PRACTITIONER LICENSED UNDER STATE LAW TO ORDER THIS PRODUCT. PRIOR TO USING THE ANCHORFAST ORAL ENDOTRACHEAL TUBE FASTENER, BE SURE TO READ THE PRODUCT INSTRUCTIONS FOR USE.

The Hollister logo and AnchorFast SlimFit are trademarks of Hollister Incorporated.
© 2019 Hollister Incorporated.

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400



Critical Care