

Restore Wound Cleanser

Technical Data Sheet

General Description

Restore Wound Cleanser aids in cleansing dermal wounds and removing particulate matter and other debris. Mild formula, free of antimicrobial agents, fragrance, and colorants. No-rinse formula.

Application

To remove exudates and debris, apply Restore Wound Cleanser on the wound and surrounding area. If necessary, allow debris to soften. No rinsing required. Use pad to remove excess debris and dry periwound skin.

Precautions: External use only. Keep out of reach of children. Avoid contact with eyes. In case of accidental ingestion, seek professional help. Consult a physician for animal bites, deep puncture wounds and serious burns, or if condition worsens and does not improve in seven days.

Physical Properties

Form:	Liquid
Type:	Surfactant solution
Color:	Colorless to a very pale yellow
pH:	6.0 – 7.0
Specific Gravity:	0.9800– 1.0700
Preserved:	Yes
Shelf Life:	2 years
Odor:	None. Fragrance free

Ingredients: Purified water, poloxamer 188, potassium phosphate, phenoxyethanol, sodium chloride, methylparaben, sodium hydroxide, ethylparaben, propylparaben, butylparaben, isobutylparaben.

Features

Mild surfactant

Non-irritating, pH balanced formula, free of antimicrobial agents

No detergents

All ingredients listed on label

Adjustable spray nozzle (Stream/Spray)

Free from animal derivatives, nut oils, fragrance and colorants

Bottle locking mechanism

Benefits

Gently removes wound debris and bacteria without harming fragile tissue

Will not compromise the natural healing process

No need to rinse

Clinicians can evaluate suitability for every patient individually

Clinician-controlled direct stream or gentle mist for cleansing, irrigation and wound hydration

Minimizes the risk of allergic reactions

Helps prevent bottle tampering

Restore Wound Cleanser

Size/Description	Stock No.	Qty/Box	HCPCS
8 oz (236 mL) pump spray bottle	529975	12	A6260
12 oz (354 mL) pump spray bottle	529976	12	A6260


hollisterwoundcare

Complex Problems. Innovative Solutions.

Hollister Wound Care LLC
1580 South Milwaukee Avenue
Suite 405
Libertyville, Illinois 60048
1.888.740.8999
Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400
www.hollisterwoundcare.com