

# Two Piece Appliances

## General Guidelines for Care

Empty pouch when it is one-third full of discharge or gas.

Change the pouch if there is a leak.

Change pouch according to directions provided by a knowledgeable healthcare provider.

For more detailed information, review instructions for use provided with the product or contact Hollister Customer Service on freecall 1800 335 911 in Australia, or 0800 167 866 in New Zealand.

Hollister offers an extensive line of ostomy products. If your current product is not working, contact your Healthcare Professional or Hollister Customer Service, who can help you.

### **CONTACT A KNOWLEDGABLE HEALTHCARE PROVIDER IF ANY OF THE FOLLOWING OCCUR**

#### **Colostomy/Ileostomy Skin irritation**

Recurrent leaks around the pouch or skin barrier

Excessive bleeding of the stoma

Blood in the stool

A bulge in the skin around the stoma

A stoma that appears to be getting larger

A stoma that appears to be getting smaller

#### **Urostomy**

Any sign of a urinary tract infection, including dark, cloudy urine,  
strong-smelling urine, fever, back pain (where kidneys are located),  
loss of appetite, nausea, or vomiting)

Skin irritation

Urine crystals on or around the stoma

Warty, discoloured skin around the stoma

Recurrent leaks around the pouch or skin barrier

Excessive bleeding of the stoma

Blood in the urine

A bulge in the skin around the stoma

A stoma that appears to be getting larger

A stoma that appears to be getting smaller

