

Premium Powder

For the management of excoriated skin

- Hollister Premium Powder protects raw and weeping skin by creating a sticky gel when in contact with body fluid.
- Premium Powder may be dispensed prior to application onto a suitable medium from which the powder may be tapped onto excoriated skin in order to meet Infection Control requirements.
- Apply sparingly to absorb moisture or exudate from skin prior to applying a secondary dressing or ostomy skin barrier.

How to Use

- Cleanse the area.
- Remove cap and squeeze gently to apply light dusting of powder.
- Remove excess.
- Apply skin barrier and pouch or secondary dressing.



Product Code 7906

