

Skin Gel Protective Dressing Wipes

Better products with you in mind

HOW TO USE

Cleanse skin with Hollister Skin Cleanser (product 7210)

Use wipe to apply Skin Gel to desired area

Skin Gel may be removed with Hollister Skin Cleanser or soap and water



Product Code 7917

- For use under all adhesive tapes and dressings, cements, NG tubes and catheters
 - Wipe over outer edge of hydrocolloid dressings to provide moisture proof seal
 - Provides a barrier from corrosive fluids and bacteria in skin folds, reducing the incidence of skin breakdown
- Keeps corrosive fluids away from the skin, reducing the incidence of skin breakdown
 - Use only on intact skin