

Hollister ETAD

Oral Endotracheal Tube Attachment Device

Application and Observation Tips



Read application instructions before applying device

1. Ensure patient skin is clean and dry.
2. Centre narrow bridge between the base of the nose and the edge of the upper lip. (This will eliminate pressure and be more comfortable for the patient).
3. Position the side panels in line with the patient's lower earlobes. (This will ensure more clearance of the device off the lip).
4. **Observe** patient for facial oedema due to metabolic state and adjust neck strap as necessary.
5. Should oedema develop around the hydrocolloid, the device should be checked immediately and changed if necessary.
6. The device may stay in place for up to 5 days, **always observing for tube security**. Change device if lifting of hydrocolloid occurs.
7. Movement of tube along the shuttle should be added to the nursing plan. Oral mouthcare is recommended at regular intervals.
8. **Observe** for weight of external lines on device and **ensure support at all times**.

NOTE:

Patients without teeth or unable to wear upper dentures may lack maxillary support required for the device.

Patients with full beard and/or moustache may lack necessary skin space to anchor barrier unless shaved.

To place your order or for further information, please contact Customer Service

