

Restore Plus

Instructions for use



Store in a cool place

Choose correct size dressing so that it extends 2.5cm beyond the wound edges on all sides

Warm the dressing (eg between hands or in pocket) this will enhance adhesion.

Remove dressing from pack

Use wound grid if desired to chart wound progress

Thoroughly rinse or irrigate the wound area if appropriate

Clean and dry the peri-wound skin to allow for secure adhesion

Do not apply solutions, lotions or creams to peri-wound area

Centre dressing over the wound site

Gently apply dressing from the centre and press the dressing to the skin to smooth out any wrinkles

Hold the dressing in place for 60 seconds to insure optimum adherence

Do not lift dressing during wear time

Code	Size	Box Qty
9956	10cm x 10cm	5
9957	15cm x 20cm	3
9958	20cm x 20cm	3
9959	Sacral	5

The dressing should be left in place (not more than 7 days) unless it is uncomfortable, leaking, there are clinical signs of infection, or until 75% of the surface area of the dressing has changed colour.

