

Skin Gel Protective Dressing Wipes

General Information

- Use to help protect the skin from stripping caused by adhesive removal.
- Apply it after the skin has been washed and thoroughly dried.
- Not recommended for use with extended wear skin barriers, such as a **Flextend** Skin Barrier, as this can interfere with adherence of the product.

Using Skin Gel Wipes

1. Wash and dry your skin thoroughly.
2. Tear open the Skin Gel Wipe package and take out the wipe. Apply to skin that will be covered by adhesive.
3. Wait until the area is **dry** to the touch.
4. Apply your ostomy pouching system as usual.

Frequently Asked Questions

Q: Is a Skin Gel Wipe always needed?

A: No, Skin Gel Wipes are an optional accessory. Use it if you need to change your skin barrier frequently to help protect skin areas that are fragile. Do not use Skin Gel or any other similar product if you are using a Flextend (extended wear) Skin Barrier.

Q: Does Skin Gel Wipe make my skin sticky so that my barrier adheres better?

A: No, it is **NOT** an adhesive. It simply adds a protective layer to the surface of your skin.

Q: Sometimes my skin stings when I apply Skin Gel, why is that?

A: Skin Gel and most other skin preps or sealants contain alcohol, which can cause a stinging sensation if applied to irritated skin.* Stop using Skin Gel until your skin has healed.

**Broken or irritated skin near the stoma should be evaluated by a healthcare professional who will help determine the cause of the skin problem and how to prevent further injury.*



Skin Gel Wipes
Stock No 7917

Universal Remover Wipes

General Information

- Helps remove adhesive skin barriers.
- Removes adhesive residue from the skin.

Using Universal Remover Wipes

1. Tear open the Universal Remover Wipe package and take out the wipe.
2. Press or rub the wipe against your skin as you peel off your skin barrier.
3. Press or rub the wipe over any areas of the skin where adhesive residue remains.
4. Wash the skin with residue-free soap and water. Rinse with water and dry thoroughly.
5. Apply your ostomy pouching system as usual.

Frequently Asked Questions

Q: Is Universal Remover always needed?

A: No, this product is an optional accessory. Use it to remove adhesive residue from your skin, or to make it easier to remove your skin barrier or tape.

Q: Why must I wash with soap and water after using Universal Remover Wipes?

A: The wipe contains a solvent. If the solvent is not removed completely from your skin, it can interfere with the adherence (stick) of the next skin barrier or cause skin irritation.



Universal Remover Wipes
Stock No 7760



Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048
1.888.740.8999

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400

www.hollister.com