

Adapt Barrier Rings and Convex Barrier Rings

Ostomy Care Tips

General Information

- Use to fill in or caulk uneven skin contours near the stoma to create a flatter surface.
- Use to prevent ostomy drainage from getting under the pouch seal.
- Can be cut, stretched and stacked together to improve the fit of your pouching system.

Using Adapt Barrier Rings or Adapt Convex Barrier Rings

1. Wash and dry your skin thoroughly.
2. Remove the protective coverings from both sides of the barrier ring and stretch or mold the ring to fit your stoma size.
3. Apply the flat side of the ring to the adhesive area of your ostomy pouching system.
4. Apply your ostomy pouch as usual.

Frequently Asked Questions

Q: How do I know what size ring to use?

A: Select the size that most closely matches the size of your stoma. The rings may be pulled and stretched to fit.

Convex Barrier Rings can be Stretched as follows:

Stock No.	Stretch up to:
79520	1" (25mm)
79530	1 3/8" (35mm)
79540	1 3/4" (45mm)

Q: What if the ring is too big for my stoma?

A: You can make the ring smaller by cutting away a section and pressing the cut ends back together.

Q: My stoma is not round. Can I still use the rings?

A: Yes! The rings can be pulled or stretched into the shape you need.

Q: How should I remove the rings?

A: The ring will come off when you remove your pouch. If residue remains, gently roll and peel it from the skin.

Q: Can the rings be used a second time?

A: No, the Adapt Barrier Rings and Adapt Convex Barrier Rings are not designed for re-use and would not adhere well if reapplied.

Q: Can I use this product when my skin is irritated?

A: Yes*, Adapt Barrier Rings and Adapt Convex Barrier Rings are alcohol-free and will not cause pain when used on irritated skin.

**Irritated skin near the stoma should be evaluated by a healthcare professional who will help determine the cause of the skin problem and how to prevent further injury.*


Hollister and logo, and Adapt are trademarks of Hollister Incorporated.
©2005 Hollister Incorporated.
907392-1005



Adapt Convex Barrier Ring being stretched to oval shape.



Adapt Convex Barrier Ring stretched to oval shape and placed on a skin barrier.

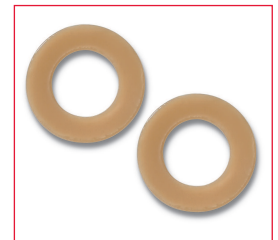

Adapt Barrier Rings
Stock No 780x.



Adapt Convex
Barrier Rings
Stock No 795xx.



Attention to Detail. Attention to Life.

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048
USA
1.800.323.4060

www.hollister.com