




ActiFlo

Indwelling Bowel Catheter System

Sustainable performance.

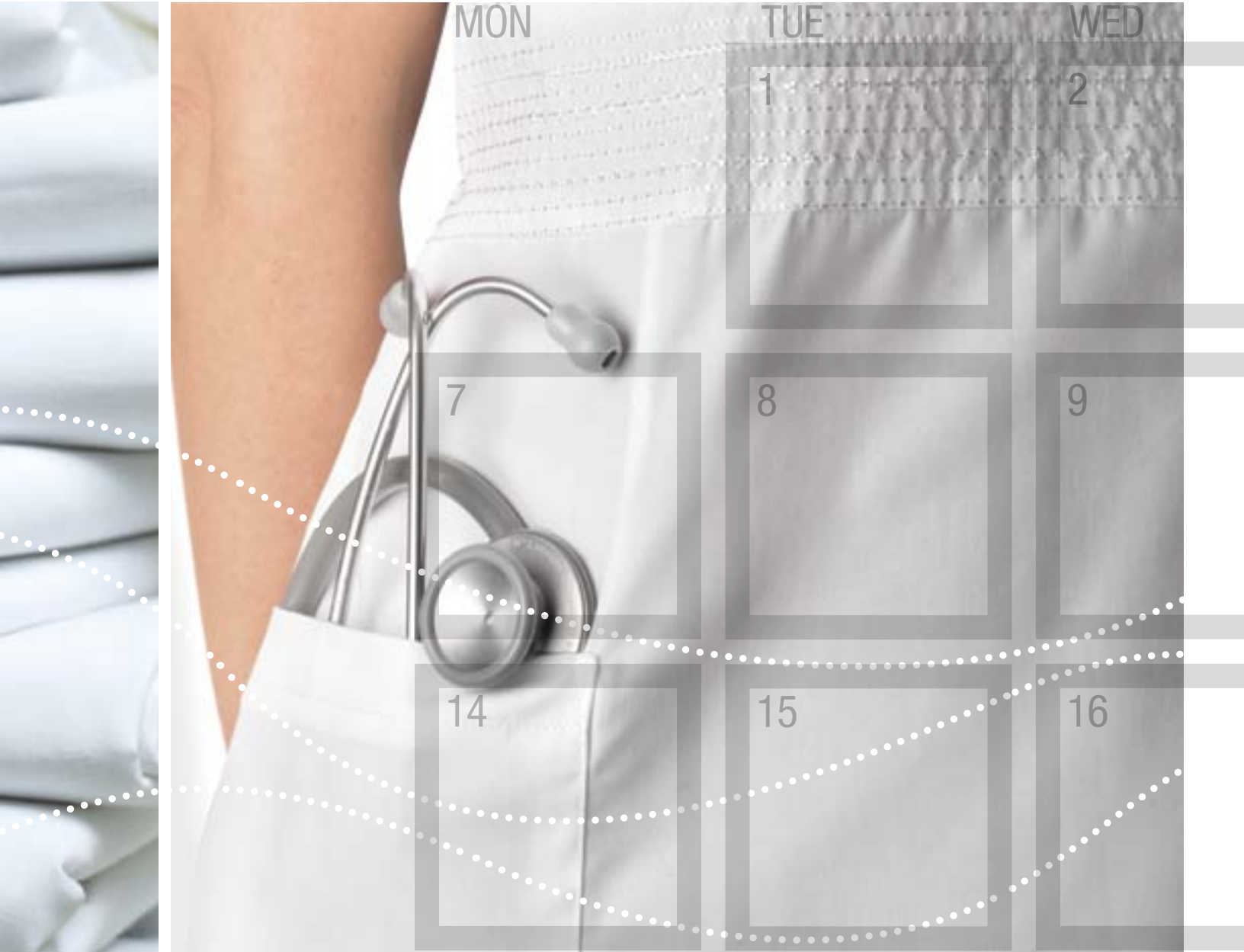

The ActiFlo System is the only bowel catheter system capable of proactively managing various stool consistencies, thereby maximizing indwell time. It is a total system that is designed to enhance overall performance, help control costs, and help improve patient outcomes.

- Effectively manages fecal incontinence to help reduce skin breakdown that can lead to the formation of pressure ulcers, and helps minimize the spread of infections related to fecal incontinence.
- The only total bowel management system designed to perform gravity irrigation to maintain patency.
- The only system that provides access for rectal medication delivery and retention.
- Facilitates stool sampling while the system remains in place.
- Collapse-resistant cylinder and low-pressure retention cuff help minimize leakage and tube expulsion.
- Enables management of stool of varying consistency.



Stop-Flow Balloon



For more information about the ActiFlo Indwelling Bowel Catheter System, contact your sales representative.

Product features

Stop-Flow Balloon

- creates smooth insertion aid
- occludes catheter for medication and irrigant retention
- helps manage various stool consistencies

Collapse-Resistant Cylinder

- helps reduce incidence of catheter occlusion
- designed to help minimize leakage and expulsion

Low-Pressure Retention Cuff

- assists with holding tube in place

Low-Impact Zone

- designed to help avoid impact on anal sphincter (available in 2 sizes, 4 cm and 6 cm)

Catheter Connectors

- provide access for colonic irrigation and rectal medication delivery
- color-coded and labeled for easy identification
- indicate inflation status via pilot balloons

Gravity Irrigation Bag

- allows for efficient irrigation

Sampling/Tube Flushing Port

- expedites hygienic stool sampling
- facilitates tube clearing

Radiopaque Marker

- identifies catheter under x-ray and fluoroscopy

Odor-Barrier Collection Bags

- help diminish odor

Coated Drain Tube

- promotes stool drainage and odor control
- low friction helps promote easy handling

Hollister Skin Barrier and Anchor Straps

- barrier adhesion gentle on skin
- help stabilize catheter to reduce internal migration
- help avoid twisting and inadvertent catheter dislodgment

Twist/Lock Connector

- connects securely to Hollister disposable or drainable odor-barrier collection bags

Adjustable Sheet Clip

- secures tube in desired position



Collapse-Resistant Cylinder and Low-Pressure Cuff




Catheter Connectors



Proactive Dependable Versatile

Stock No	Catheter Kit includes	Stock No	Catheter Kit includes
32004 (4 cm) and	Catheter (1)	32006 (4 cm) and	Catheter (1)
	3000 mL Drainable Odor-Barrier Collection Bag (1)		2000 mL Disposable Odor-Barrier Collection Bags (2)
	Hydrocolloid Skin Barriers (2)		Hydrocolloid Skin Barriers (2)
	Lubricating Jelly Packets (2)		Lubricating Jelly Packets (2)
32005 (6 cm)	Syringe (1)	32007 (6 cm)	Syringe (1)
	1000 mL Irrigation Bag (1)		1000 mL Irrigation Bag (1)
	Instruction Booklet (1)		Instruction Booklet (1)

Stock No	Accessories	Quantity/Box
31003	3000 mL Drainable Odor-Barrier Collection Bag	5
31008	2000 mL Disposable Odor-Barrier Collection Bag	5
31005	1000 mL Irrigation Bag	30

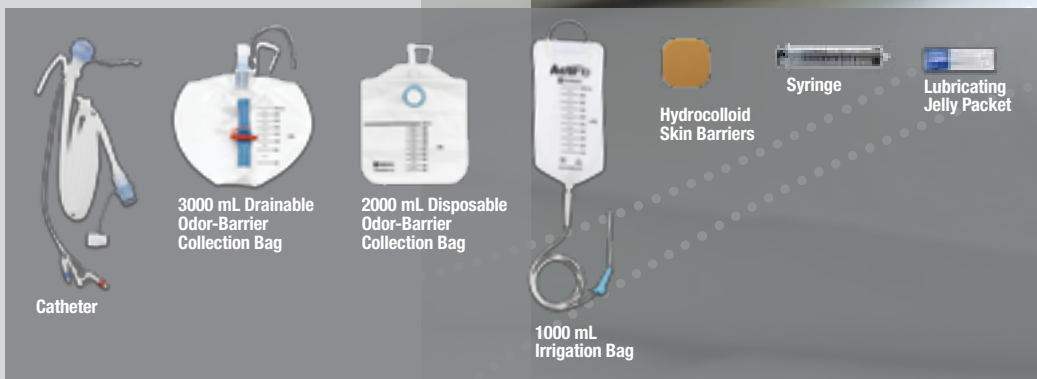

Hollister Incorporated
Libertyville, IL USA

For detailed clinical questions concerning our products:
1.888.740.8999
For orders only: 1.800.323.4060

Distributed by
Hollister Limited
Aurora, Ontario
1.800.263.7400

www.Hollister-ActiFlo.com

The approved labeling for this device states it is not intended for use longer than 29 days.

CAUTION: Federal (USA) Law restricts this device to sale by or on the order of a physician. Prior to use of the ActiFlo Indwelling Bowel Catheter System, be sure to read the entire ActiFlo Indwelling Bowel Catheter System Instructions for Use package insert supplied with the product for device Intended Use, Description, Contraindications, Warnings, Precautions, Adverse Events, and Instructions for Use.

Hollister and logo, Hollister and ActiFlo are trademarks of Hollister Incorporated. Covered under one or more of the following patents: US Patents 7147627, 5569216, 7722583, Australia Patents 2003207608, 20070201015, 2000274575, Canada Patents 2462570, 2421405, Japan Patents 4260107, and other patents pending.

(c) 2010 Hollister Incorporated. Printed in USA.

911120-710