

Restore Hydrogels

Technical Data Sheet

General Description

Restore Hydrogel Dressings promote a moist environment in a variety of wounds including Stages II-IV pressure ulcers, diabetic skin ulcers, venous ulcers, 1st and 2nd degree burns, skin tears, cuts, abrasions, and conditions associated with peristomal care. Non-greasy, non-staining, and fragrance free.

Application

Cleanse the wound with Restore Wound Cleanser or other appropriate solution. Apply Restore Hydrogel dressing to wound and cover with a secondary dressing. Secure secondary dressing with tape or other appropriate materials. Change the Restore Hydrogel Dressing every 24 to 72 hours or as required to maintain a moist environment.

Precautions: External use only. Keep out of reach of children. Avoid contact with eyes. In case of accidental ingestion, seek professional help. If condition worsens or does not improve within seven days, consult a physician.

Physical Properties

Form: Amorphous gel, impregnated
4 inches x 4 inches gauze and
2 inches x 3.5 yards strips
Clarity: Clear
pH: 4.5 – 6.0

Ingredients: Deionized water, glycerin USP 99.7%, sodium polyacrylate, propylene glycol USP, hyaluronic acid, sodium metabisulfite FCC, methylparaben NF, propylparaben NF.

Features

Provides a moist wound environment

Change Restore Hydrogel dressing every 24 to 72 hours or as required to maintain a moist environment

Available in three application forms: clear gel, sterile gauze sponge, sterile gauze strips

Ingredients fully disclosed on label

Formulation is clear

Does not contain nut oils, animal derivatives, fragrance, and botanical extracts

Benefits

A moist healing environment is essential to the natural healing process

Fewer dressing changes = more cost-effective care

Provides clinical flexibility by accommodating various wound types

Clinicians can evaluate suitability for every patient individually

Caregiver can view wound bed even when Restore Hydrogel dressing is present

Minimizes the potential for allergic reactions

Restore Hydrogel Dressings

Size/Description	Stock No	Qty/Box
Impregnated Gauze Sponge 12-ply 4 inch x 4 inch, Sterile HRI #08380-0099-72	9972	15
Impregnated Gauze Strip 2 inch x 3.5 yard, Sterile HRI #08380-0099-73	9973	12
Amorphous Gel 3 ounce Tube, Non-Sterile HRI #08380-0099-74	9974	12



Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1-800-323-4060

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1-800-263-7400