

Restore

Wound Cleanser

Technical Data Sheet

General Description

Restore Wound Cleanser aids in cleansing dermal wounds and removing particulate matter and other debris. Mild formula, free of antimicrobial agents, fragrance, and colorants. No-rinse formula.

Application

To remove exudates and debris, apply Restore Wound Cleanser on the wound and surrounding area. If necessary, allow debris to soften. No rinsing required. Use pad to remove excess debris and dry periwound skin.

Precautions: External use only. Keep out of reach of children. Avoid contact with eyes. In case of accidental ingestion, seek professional help. Consult a physician for animal bites, deep puncture wounds, and serious burns, or if condition worsens and does not improve in seven days.

Physical Properties

Form:	Liquid
Type:	Surfactant solution
Color:	Colorless to a very pale yellow
pH:	6.0 – 7.0
Specific Gravity:	0.9800 – 1.0700
Preserved:	Yes
Shelf life:	2 years
Odor:	None. Fragrance free

Ingredients: Purified water, poloxamer 188, potassium phosphate, phenoxyethanol, sodium chloride, methylparaben, sodium hydroxide, ethylparaben, propylparaben, butylparaben, isobutylparaben.

Features

Benefits

Mild Surfactant	Gently removes wound debris and bacteria without harming fragile tissue
Non-irritating, pH balanced formula, free of antimicrobial agents	Compatible with the natural healing process Meets the requirements of ISO 10993, Part 5, for Cytotoxicity
No detergents	No need to rinse
All ingredients listed on label	Clinicians can evaluate suitability for every patient individually
Adjustable spray nozzle (Stream/Spray)	Clinician-controlled direct stream or gentle mist for cleansing, irrigation, and wound hydration
Free from animal derivatives, nut oils, fragrance, and colorants	Minimizes risk of allergic reactions

Restore Wound Cleanser

Size/Description	Stock No	Qty/Box
8 ounce trigger spray bottle	9975	12
12 ounce trigger spray bottle	9976	12



Attention to Detail. Attention to Life.

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1-800-323-4060

Distributed in Canada by
Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1-800-263-7400