

Hydrofera**BLUE**

Ready. Set. Go.

Simply set in place
and secure;
no hydration required.

Wound Site Preparation



Cleanse the wound bed in accordance with established procedures.

Select Size and Apply



Select the appropriate dressing size to ensure the dressing will cover the entire wound. Apply with the foam side towards the wound, and the shiny side with the thin film up.

Secure



Secure the dressing using a bandage or tape.

Hydrofera Blue Ready foam can be used in conjunction with compression wraps and total contact casting.

Wear Time



Dressing can be left in place up to 7 days, depending on the exudate level and clinical conditions of the wound.

