

Adapt Adhesive Remover Wipe

General Information

- Helps remove tapes, barriers and cements
- Removes adhesive residue from the skin
- Silicone-based formulation
- No sting from alcohol

Using Adhesive Remover Wipes

1. Tear open the package and take out the wipe.
2. Press or rub the wipe against the skin as you peel off the adhesive.
3. Press or rub the wipe over any areas of the skin where tape residue remains.
4. Gently wipe off any remaining residue.



Adapt Adhesive Remover Wipe
Stock No 7760



Wipes - Rub the wipe over skin as you peel off the adhesive.



Wipes - Clean off remaining tape residue.

FREQUENTLY ASKED QUESTIONS

<p>Is an adhesive remover always needed?</p>	<p>No, this product is optional. Use it to remove adhesive residue from the skin, or to make it easier to remove tape.</p>
<p>Will this be uncomfortable to use?</p>	<p>This silicone-based product is formulated to be a no-sting alternative. When used correctly, the adhesive remover will help to make the removal of adhesive residue easier and help reduce the potential for skin stripping.</p>
<p>Do I need to wash the adhesive remover off of the skin?</p>	<p>The adhesive remover formulation evaporates quickly and dries without residue, so the next adhesive can attach securely. If you wash the skin after use be sure to rinse and dry thoroughly before applying another adhesive.</p>

Routine follow-up with your healthcare professional is recommended.

Instructions for Use: Alcohol free – No Sting! For external use only. Removes tapes, barriers, cements and their adhesive residue from the skin. Open packet and immediately begin using the wipe. Starting at the edge, gently peel back the skin barrier or adhesive as you apply the Adapt Adhesive remover at the junction between the adhesive and the skin. Continue to wipe and peel the barrier or adhesive back until it has been completely removed. When the barrier or tape is completely removed from the skin, gently wipe off any remaining residue. Store in a cool, dry place.

WARNING: Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

FLAMMABLE: No smoking. Do not use near flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition.

CE Not all products are CE marked.

The Hollister logo, Adapt and “Healthy skin. Positive outcomes.” are trademarks of Hollister Incorporated.
© 2020 Hollister Incorporated.

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com