

Adapt No Sting Skin Protective Wipe

General Information

- Use to help protect the skin from bodily waste and stripping associated with removal of adhesive
- Delivers a no sting application which may be associated with products that contain alcohol
- Skin protective wipes are not recommended under extended wear skin barriers as this may decrease wear time



No Sting Skin Protective Wipes
Stock No 7917

Using No Sting Skin Protective Wipe

1. Wash and dry your skin thoroughly.
2. Tear open the skin protective wipe package and take out the wipe.
3. Apply directly to skin area to be protected.
4. Wait until the area is completely dry to the touch.
5. Apply your ostomy pouching system or adhesive as usual.

FREQUENTLY ASKED QUESTIONS

<p>Are skin protective wipes always needed?</p>	<p>No. Skin protective wipes are optional. Use to help protect the skin from bodily waste and stripping associated with removal of adhesive. A skin protective wipe may be used when removing adhesive/tape frequently.</p>
<p>Can skin protective wipes be used with extended wear skin barriers?</p>	<p>It is not recommended to use skin protective wipes or any other similar product if you are using an extended wear skin barrier as this may decrease wear time.</p>
<p>Do Adapt skin protective wipes help to minimize skin-related issues?</p>	<p>No sting skin protective wipes are a gentle, water-based alternative developed to minimize the irritation and pain that can occur with wipes that contain alcohol. They also offer effective skin protection to help reduce skin stripping associated with repeated application and removal of the adhesive or skin damage from bodily fluids.</p>
<p>Does the skin protective wipe allow the adhesive to adhere better?</p>	<p>No, it is not an adhesive. It simply adds a protective layer to the surface of your skin.</p>

Adapt No Sting Skin Protective Wipe

Additional Instructions:

Routine follow-up with your healthcare professional is recommended.

Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to ease removal of tapes, barriers, cements and their adhesive residue from the skin.

Warnings: To help reduce the potential for infection and/or other complications, do not reuse.

EXTREMELY FLAMMABLE: Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed. Precautions for external use only. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Instructions For Use: Skin should be clean and dry prior to application. Open packet and immediately begin using the wipe. Starting at the edge, gently peel back the skin barrier or adhesive as you apply the Adapt universal remover at the junction between the adhesive and the skin. Continue to wipe and peel the barrier or adhesive back until it has been completely removed. When the barrier or tape is completely removed from the skin, gently wipe off any remaining residue. Please follow all local disposal regulations. Store in a cool, dry place.

CE	Not all products are CE marked.
----	---------------------------------

The Hollister logo, Adapt and “Healthy skin. Positive outcomes.” are trademarks of Hollister Incorporated.
© 2020 Hollister Incorporated.

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com