Guide for Use


1 Wash hands with mild soap and water.


2 Fully fold back the packaging by peeling down on the clear side of the package.

3 Remove the blue cap from the protective tip of the catheter. Advance the catheter into the protective tip. Do not advance the catheter past the end of the protective tip.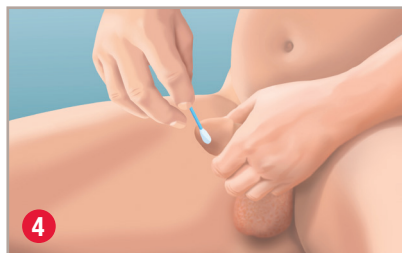


4 Hold the penis. Pull back the foreskin (if present) and cleanse around the opening of the urethra and then the glans with mild, unscented soap and water or an unscented wet wipe. If you are using the Apogee Plus kit, use the PVP swabs provided for cleansing.

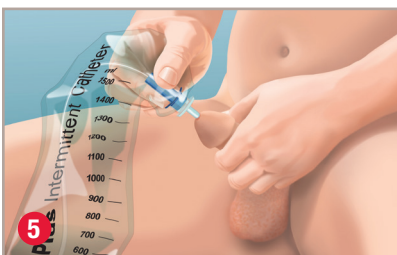5 With one hand, hold the penis in an upward position at the base of the glans. With the other hand, grasp the catheter below the base of the protective tip and insert the protective tip into the urethral opening. Advance the catheter until urine starts to flow.

6 When the urine flow stops, slowly pull the catheter out to capture any urine remaining in the bladder.

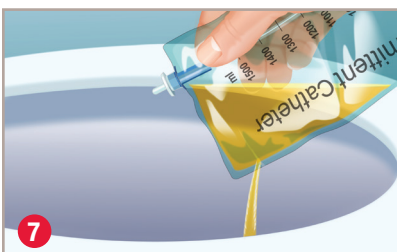7 To empty before disposing the product, tear the bag at the perforation and pour urine into a toilet or other collection device.

8 The catheter may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash hands with mild soap and water.

Watch the Apogee Plus male instructional video:
www.hollister.com/apmale

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

- Ready-to-use, pre-lubricated, single use closed system intermittent catheter
- Not made with natural rubber latex
- Protective tip
- 2 off-set smooth catheter eyelets
- Color-coded funnel
- 1500 mL collection bag

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

PRECAUTION: Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

Rx Only. Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

Hollister Incorporated
 2000 Hollister Drive
 Libertyville, Illinois 60048
 1.800.323.4060

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