
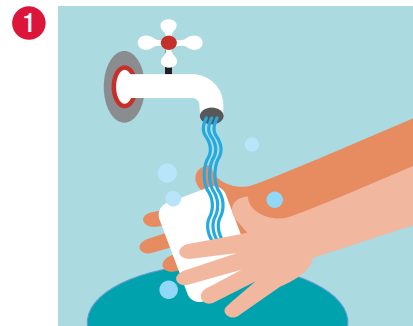
Boys' Guide for Use


Onli
Ready to Use Hydrophilic Intermittent Catheter

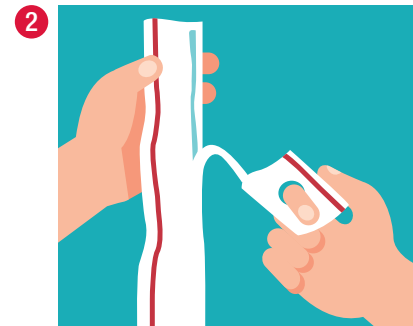1 Wash your hands with soap and water.

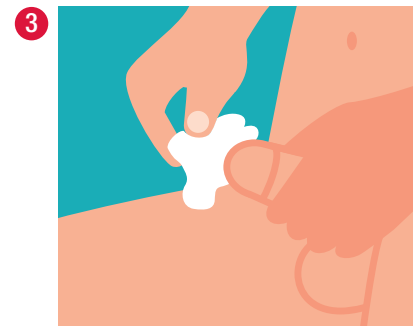
2 Open the package by pulling the finger hole toward you and at least half way down.


3 Hold your penis, and pull back the foreskin (if you have one). Clean the tip of your penis with soap and water, or a cleansing wipe.

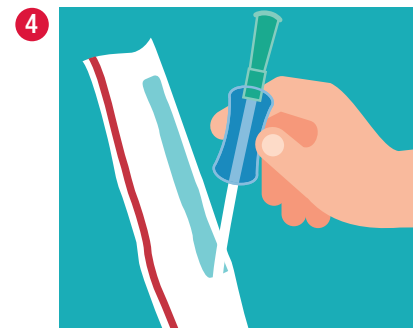Hint: Don't squeeze your penis too hard. This can block the opening and make it difficult to insert the catheter.

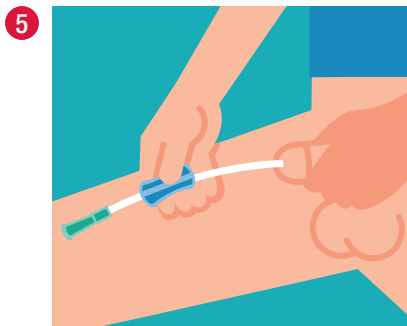
4 Hold blue gripper to remove catheter from package

5 With one hand, hold your penis facing up. With the other hand, pull gripper away from the funnel. You can hold the gripper anywhere along the catheter so you do not touch the catheter.

Hold gripper tightly and begin inserting catheter into the tip of your penis. Slightly release the gripper, and slide it back toward the funnel. Keep inserting the catheter until pee starts to flow. Be sure the open end of your catheter is directed into the toilet.

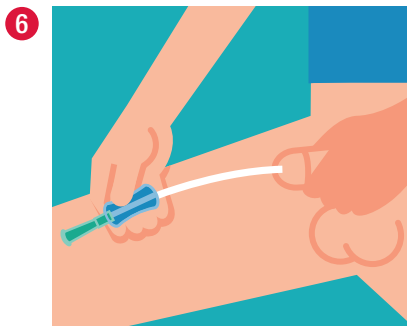
When the pee stops, slowly pull the catheter out. Pee may start to flow a little bit more. That is normal. Just pause and wait for pee to stop again before removing the catheter.


6 Another option: Keep the gripper connected to the funnel. With one hand, hold your penis facing up. With the other hand, hold the catheter down at the gripper/funnel end. Insert the catheter into the tip of your penis until pee starts to flow.

Be sure the open end of your catheter is directed into the toilet.

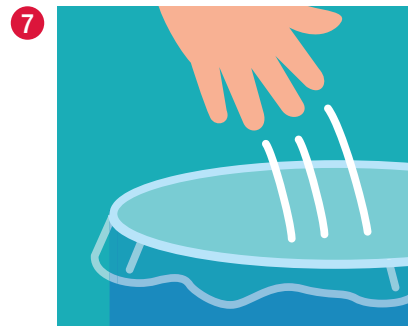When the pee stops, slowly pull the catheter out. Pee may start to flow a little bit more. That is normal. Just pause and wait for pee to stop again before removing the catheter.

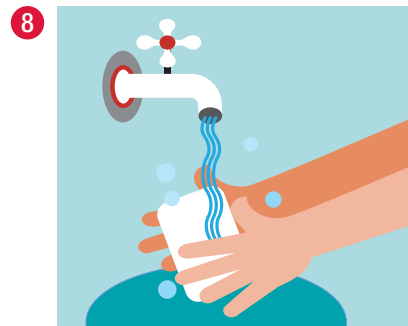7 Throw the catheter away in the garbage. Do not flush it down the toilet.

8 Wash your hands again with soap and water.

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

- Ready to use, single-use intermittent catheter
- Hydrophilic-coated, phthalate-free catheter
- 2 smooth catheter eyelets
- Color-coded funnel gripper on 16" catheter to aid insertion
- Not made with natural rubber latex

WARNING: To help reduce the potential for infection and/or other complications, do not reuse.

WARNING: If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present: severed urethra • unexplained urethral bleeding • Pronounced stricture • False passage • Urethritis – inflammation of the urethra • Prostatitis – inflammation of the prostate gland • Epididymitis – inflammation of the epididymis (testicle tube).


PRECAUTION: Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

CAUTION: Rx ONLY. Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product. Prior to use of this device, be sure to read (i) the complete information on how to use this device including Warnings, Cautions, and Instructions for Use, and (ii) all other package inserts and labels supplied with the product.

NOTE: Store boxes in a flat position and at normal room temperature.

This product consists of: thermoplastic gripper (16" catheter only) polyvinylchloride funnel and tubing not made with phthalates.

The Hollister logo and Onli are trademarks of Hollister Incorporated. © 2017 Hollister Incorporated. 923102-117

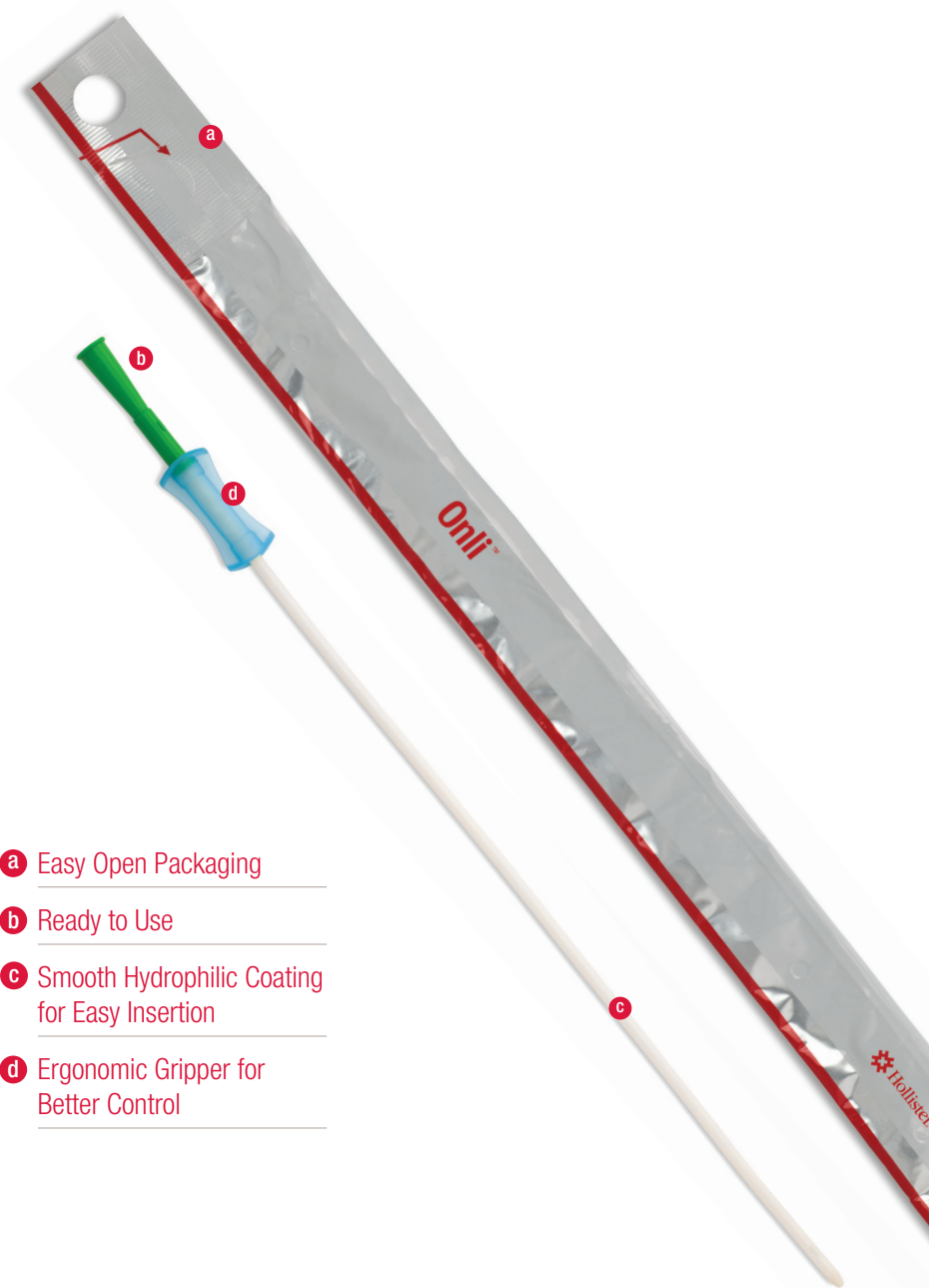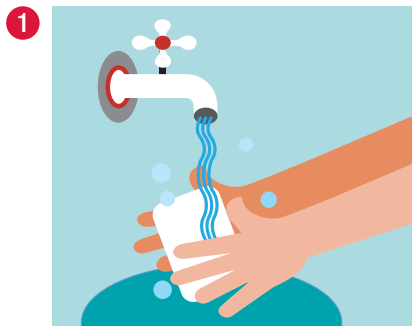
- a Easy Open Packaging
- b Ready to Use
- c Smooth Hydrophilic Coating for Easy Insertion
- d Ergonomic Gripper for Better Control

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