# User Guide Advance Plus™ Catheters

1. Wash your hands.

2. Fully fold back the packaging by peeling the clear side of the package.


3. Remove the cap from the protective tip.


4. Hold the gel reservoir in one hand and with the other hand move the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip.



5. Spread the labia and cleanse around the opening of the urethra with mild, unscented soap or a non-alcoholic wet wipe.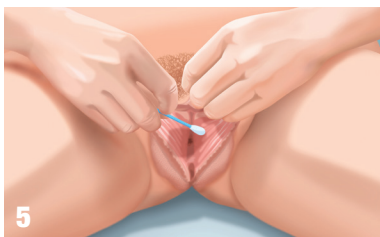

6. Continue holding the labia apart with two fingers. Holding the gel reservoir gently, insert the protective tip until the base comes in contact with the urethral opening. Keep the protective tip in place. Release the labia.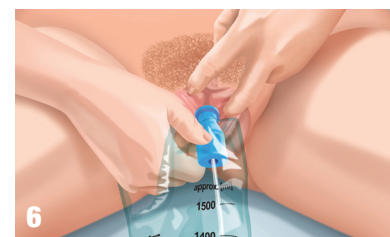

7. The gel reservoir should be held gently in place until the catheter has passed through the urethra and into the bladder. As you advance the catheter and the pouch begins to gather, if you want to pull back the pouch, hold the catheter firmly through the gel reservoir. This action will help prevent the catheter from slipping out of the urethra before you gently pull back on the pouch. Continue to insert the catheter forward until the urine starts to flow. Once urine has stopped flowing, slowly withdraw the catheter.

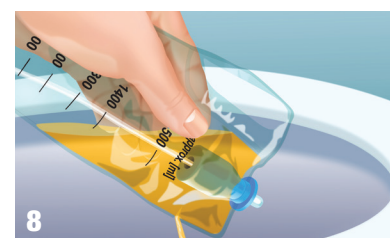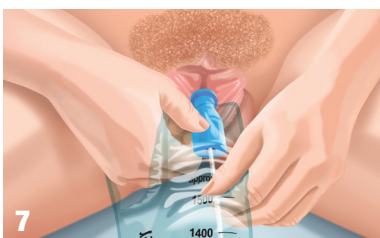
8. To empty before disposing, tear at notch to drain urine.


9. The catheter may be disposed of in a waste basket. DO NOT FLUSH DOWN TOILET. Wash hands with mild soap and water.

**Advance Plus**  
Intermittent Catheter

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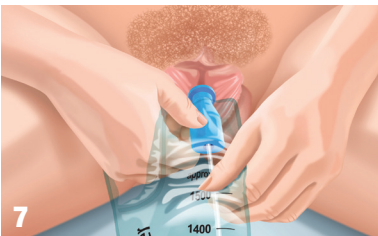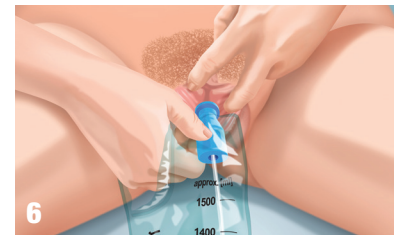**Advance Plus**  
Intermittent Catheter

**WARNING: To help reduce the potential for infection and/or other complications, do not reuse.**

**WARNING:** If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

- Ready-to-use, pre-lubricated, single-use closed system intermittent catheter
- Protective tip and collection bag supports no-touch insertion technique
- Gel reservoir
- 2 smooth catheter eyelets
- Color-coded funnel
- 1500 mL collection pouch

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric (paediatric) patients who need to drain urine from the bladder.

**PRECAUTION:** Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Pronounced stricture
- Urethritis – inflammation of the urethra
- Epididymitis – inflammation of the epididymis (testicle tube)
- Unexplained urethral bleeding
- False passage
- Prostatitis – inflammation of the prostate gland

**PRECAUTION:** Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

**NOTE:** Store boxes in a flat position and at normal room temperature.

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