

## InView™ Silicone Male External Catheter

### General Information

The male external catheter, self-adhering and engineered with a clear, silicone\* material, is available in three styles: Standard, Special, and Extra. External catheters provide a good alternative to indwelling catheters, diapers or other absorbent products.

Five sizes are available, and the packaging is color-coded for easy identification.

Small	(25 mm)	Green
Medium	(29 mm)	Purple
Intermediate	(32 mm)	Tan
Large	(36 mm)	Blue
Extra Large	(41 mm)	Orange

### Selecting the Proper Size and Preparation

- Use the color-coded Hollister sizing guide to determine correct size needed.<sup>1</sup>
- Place the color-coded Hollister sizing guide beneath the widest diameter of the penis shaft (not the glans).
- In cases of shorter shaft length, select the **InView** Special catheter for better adhesion.
- When a larger adhesive band is needed to improve adhesion, select the **InView** Extra catheter.
- Trim pubic hair if necessary.
- Wash penis with mild soap and warm water. Dry thoroughly.

### Instructions for Use

1. Open package at perforation. Remove sheath (catheter) from plastic insert, if present.
2. Place rolled end over the end of the penis, leaving a small space between the end of the penis and the cone of the sheath (catheter). If applying to an uncircumcised penis, allow foreskin to remain in its natural position. Take care not to pinch or retract the foreskin.
3. Slowly unroll onto the penis with as little wrinkling as possible until product is fully extended.
4. Gently squeeze to properly seal adhesive to the skin. Do not attempt to reposition the product. If possible, avoid leaving a rolled 'collar' around the base of the penis.






To connect this product to a drainage system, refer to the instructions for use packed with that product.

### To remove and dispose

5. Disconnect from the drainage system. Gently roll the sheath (catheter) forward and off the penis. If necessary, apply a warm wet compress (such as a wet washcloth) around the sheath (catheter) to help loosen the adhesive. Please follow all local disposal regulations

\*Not made with natural rubber latex

<sup>1</sup>Sizing guides can be ordered free of charge using #911174 or digital sizing guide can be printed from the product page on Hollister.com

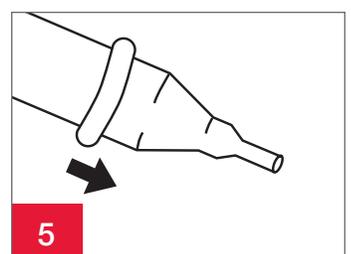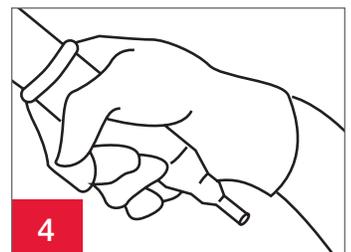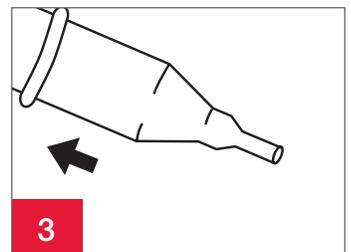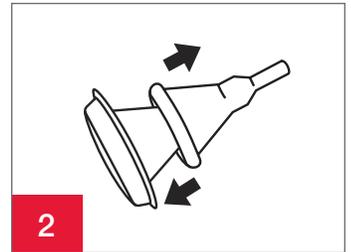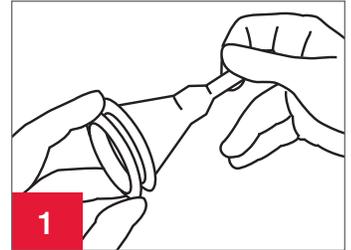

**InView™ Silicone Male External Catheter****FREQUENTLY ASKED QUESTIONS**

How long do these catheters stay on?	A variety of factors such as application technique, individual anatomy and condition of the skin may influence wear time. For good hygiene, change catheter daily.
Is sizing important in selecting the right external catheter?	Yes. The Hollister sizing guide is available to help in proper catheter size selection. If the patient is between sizes, select the larger of the two sizes, apply the catheter and crimp the excess silicone material on the shaft so the urine does not channel beneath the catheter and leak.
Is the <b>InView Silicone Male External Catheter Special</b> only for men with a short shaft?	No. Some users simply prefer an external catheter with a shorter sheath. The adhesive of the <b>InView Silicone Male External Catheter Special</b> is adequate to hold the catheter. For some men, a shorter sheath is a matter of preference or comfort.
How should I clean the bedside collector or leg bag used with an external catheter?	Between uses, empty and rinse bags with lukewarm water or clean per manufacturer instructions and/or healthcare provider recommendations. Hang to dry away from heat. Wipe outside of bag and tubing with damp cloth as needed. Clean the connector with an alcohol wipe before each use. Discard when visibly damaged or stained.
What is the best way to remove the external catheter? (Fig. 5)	Gently roll the sheath (catheter) forward and off the penis. Use of a warm, wet washcloth, held over the catheter for approximately 30 seconds may help weaken the adhesive for easier removal in some cases. Adhesive remover wipes may also be helpful.
Can external catheters be used on an uncircumcised penis?	Yes. Apply the catheter with the foreskin in its natural position.

**Indications and Intended Use**

The product is an accessory intended to channel urine from the penis to extension tubing and urinary collection bags for adult males with urinary incontinence.

**WARNINGS**

Reuse of a single use product may create potential risk to the user such as poor adhesion, leakage, and skin irritation.

**Precautions**

Do not use on irritated or compromised skin.

Do not use if allergic reaction occurs.

Ensure proper fit as instructed. If the sheath (catheter) size is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.

Follow directions to remove if experiencing swelling, numbness, discomfort, pain, discoloration, or abnormal appearance.

**Notes**

For good hygiene, change sheath (catheter) daily.

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use lotions, powders, or ointments on the skin as these products may interfere with adhesion of the product. Contact a healthcare practitioner for help with the product or with skin irritation.

Hollister, the Hollister logo and InView are trademarks of Hollister Incorporated.

© 2021 Hollister Incorporated.

911229-0121 US-01326

**Hollister Incorporated**

2000 Hollister Drive  
Libertyville, Illinois 60048  
1.800.323.4060

[www.hollister.com](http://www.hollister.com)