
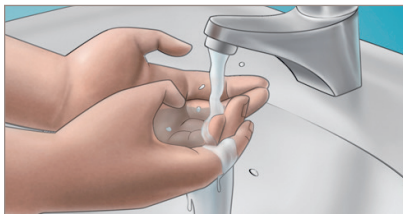
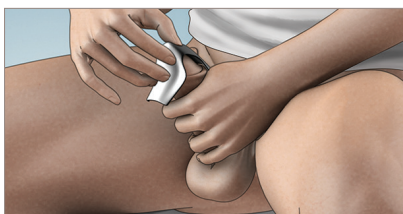
Guide for Use


1 Wash hands.

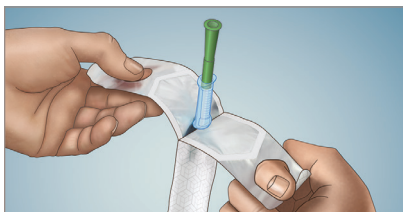2 Use mild, unscented soap or a non-alcoholic wet wipe to cleanse glans and opening of urethra as instructed by your healthcare practitioner. Choose a position to catheterize that is most comfortable for you (standing, sitting on the toilet, in wheelchair).

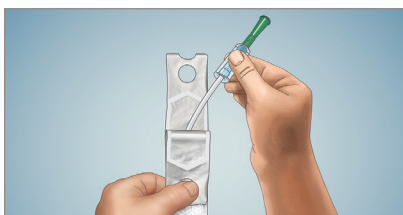
3 Using finger holes peel open the package until the funnel and gripper are visible.


4 Hold funnel and/or gripper to remove catheter.

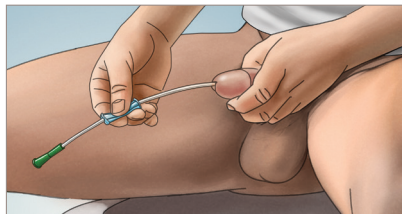5 Pull back foreskin if present.

6A Pull gripper away from the funnel. Hold gripper anywhere along the catheter, so you do not touch the catheter. With the penis stretched and in upright position, hold gripper tightly to begin inserting catheter. Slightly release gripper, and slide it back towards funnel. Then hold gripper tightly again to advance the catheter. When using a Tiemann/Coudé tip catheter, locate the notch on the funnel, which is aligned with the upward direction of the Tiemann/Coudé tip. Insert the catheter tip into the urethra with the notch in the 12 o'clock position, unless instructed otherwise by your healthcare practitioner. Repeat until urine starts to flow. Relax foreskin, if present.

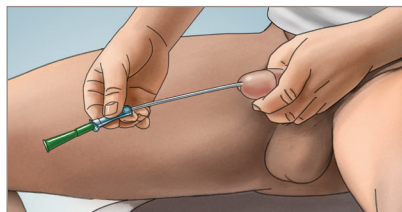6B Alternative insertion option: keep the gripper 'docked' with the funnel. With penis stretched and in upright position, hold gripper/funnel and insert catheter into the urethra with one hand. When using a Tiemann/Coudé tip catheter, locate the notch on the funnel, which is aligned with the upward direction of the Tiemann/Coudé tip. Insert the catheter tip into the urethra with the notch in the 12 o'clock position, unless instructed otherwise by your healthcare practitioner. Advance catheter until urine starts to flow. Relax foreskin, if present.

7 Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely. Catheter may be disposed of in a waste bin. **DO NOT FLUSH DOWN TOILET.** Please follow all local disposal regulations.

8 Wash hands.

For more information visit
Hollister.com/OnliCoude

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

WARNINGS

Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

Notes: Store boxes in a flat position and at normal room temperature. Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

USA: Rx Only.

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

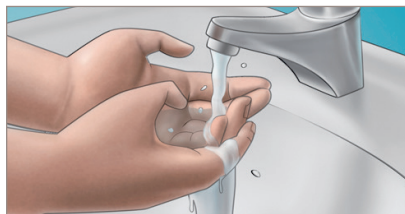
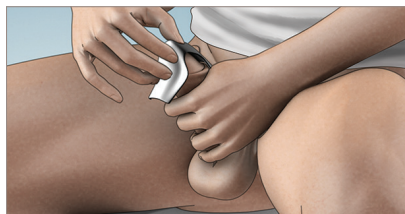
Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.888.808.7456

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