

Male Urinary Pouch External Collection Device

Introduction

This is a skills checklist for the application, and removal of the Male Urinary Pouch. The pouch is intended to collect and contain urine for individuals with urinary incontinence. The pouch is most suitable for non-ambulatory males with urinary incontinence. It is designed to be emptied and should be attached to a drainage bag. The suitability of the male urinary pouch must be assessed for each patient. (See the section below entitled "Precautions" for factors to consider.)

Date Completed	Name	Employee Number	Dept.
----------------	------	-----------------	-------

Criteria (<u>Refer to Instructions for Use for detailed information</u>)	Eval Method*	Completed
Selecting the Proper Size and Preparation		
1. Wash genital area with a mild soap and water.		
2. Trim the pubic hair. Rinse and dry thoroughly.		
3. If desired, wipe the skin that will be in contact with the skin barrier with a skin protective wipe (not included). Please refer to the instructions for use packed with that product.		
4. Be sure the drain cap is placed on the pouch drain.		
5. If necessary, cut an opening in the skin barrier slightly larger than the penis.		
Instructions for Use		
1. Remove the protective release paper from the skin barrier.		
2. Place the penis in the opening of the pouch and press the barrier securely on skin. Avoid wrinkling, which can create channels for leakage. Gently hold the barrier material against the skin so that the user's body heat will soften the barrier material and enhance its adhesive properties. If desired, apply microporous adhesive strips (not included) around the barrier edges for an added sense of security. Please refer to the instructions for use packed with that product.		
3. Remove the drain cap at the bottom of the pouch and attach the tubing to a drainage bag (not included).		
To Remove and Dispose		
1. To dispose, please follow all local disposal regulations.		

* Evaluation Method: (L) return demos live in class or skill lab, (O) observation in worksetting, (CR) chart review, (V) verbalizes information, (R) read only – not performed – for information, (NA) not applicable.

Reviewed by (signature) _____ .

Warnings

- Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilization may compromise the product. This may result in odor or leakage.

Precautions

Discontinue use of this product if any of the following signs or symptoms occur:

- Changes in the skin color of the penis
- Extreme skin temperature changes
- Swelling, burning, itching, pain or loss of sensation

Notes

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Frequent removal of a Flexextend™ skin barrier in a short period of time (e.g., multiple times a day) may result in skin damage. Contact a healthcare practitioner for help with continence care products.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions and Instructions.

Hollister, the Hollister logo and Flexextend are trademarks of Hollister Incorporated.

© 2021 Hollister Incorporated. US-01645

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

