## Guide for Use

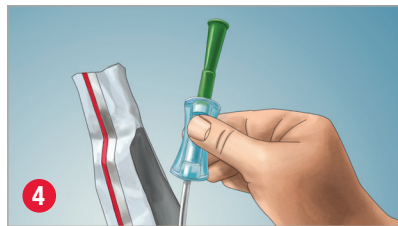
1 Wash hands with mild soap and water.


2 Use mild, unscented soap or non-alcoholic wet wipe to cleanse glans and opening of urethra.


3 Open package by pulling the finger hole toward you.

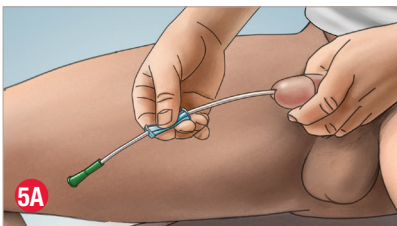4 Hold funnel or gripper to remove catheter from package.

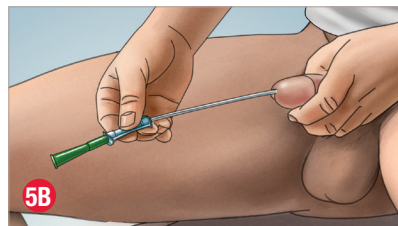
5A Pull back foreskin if present. Pull gripper away from the funnel. Hold gripper anywhere along the catheter, so you do not touch the catheter. With the penis upright, hold gripper tightly to begin inserting catheter. Slightly release gripper, and slide back towards funnel. Then hold gripper tightly again to advance the catheter. Repeat until urine starts to flow. Relax foreskin, if present.

5B Alternative insertion option: keep the gripper "docked" with the funnel. With penis upright, hold gripper/funnel and insert catheter into the urethra with one hand until urine starts to flow. Relax foreskin, if present.

Once urine flow has stopped, slowly withdraw the catheter to ensure bladder is completely empty. Gently remove catheter.

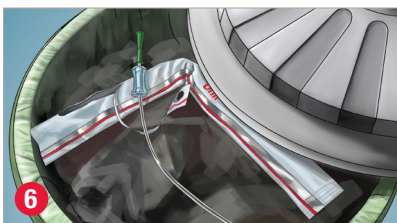Once urine flow has stopped, slowly withdraw the catheter to ensure bladder is completely empty. Gently remove catheter.

6 The catheter may be disposed in a waste bin. Do not flush catheter or package down the toilet.

7 Wash hands with mild soap and water.

Watch the Onli male instructional video:  
[www.hollister.com/en/onlimale](http://www.hollister.com/en/onlimale)

**WARNING:** To help reduce the potential for infection and/or other complications, do not reuse.

**WARNING:** If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

- Ready to use, single-use intermittent catheter
- Hydrophilic-coated, phthalate-free catheter
- 2 smooth catheter eyelets
- Color-coded funnel
- Not made with natural rubber latex
- Gripper on 16" catheter to aid insertion and support no-touch insertion technique

This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and pediatric patients who need to drain urine from the bladder.

**PRECAUTION:** Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

**PRECAUTION:** Self-catheterization should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization frequency varies by person, the recommended frequency of your catheterization should be provided by your healthcare practitioner. For any other questions about your catheterization, please contact your healthcare practitioner.

**Rx Only.** Federal (USA) Law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product. Prior to use of this device, be sure to read (i) the complete information on how to use this device including Warnings, Cautions, and Instructions for Use, and (ii) all other package inserts and labels supplied with the product.

**NOTE:** Store boxes in a flat position and at normal room temperature.

This product consists of: thermoplastic gripper (16" catheter only), polyvinylchloride funnel and tubing not made with phthalates.

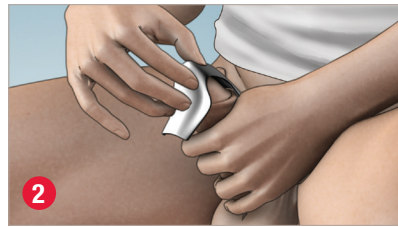
Hollister Incorporated  
2000 Hollister Drive  
Libertyville, Illinois 60048  
1.800.323.4060

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©2019 Hollister Incorporated.