


# De-moso Pouching Sistèm

## Kolekte Pwovizyon

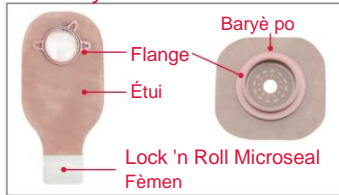

Rasanble sak, baryè po, sak jete, tisi ak/oswa twal pou netwaye, ak dlo. Si ou vle: sizo, plim, gid pou mezire, ak/oswa Pwodwi pou Telefòn.

## Retire itilize Pouching sistèm



Ak anpil atansyon leve yon kwen nan adezif la epi kale anba, dousman pouse po a lwen baryè po a. Byen jete sak itilize ak baryè po. Pa jete twalèt la.

## Netwaye ak PO sèk



Netwaye po a ak dlo oswa ak savon dou jan pwofesyonèl swen sante ou rekòmande. Rense epi sèk byen. Pa sèvi ak lwil, poud, losyon, oswa savon kalite losyon sou po a.


## Tcheke Kondisyon Po



Tcheke pou asire w ke po alantou stoma ou a (peristomal) entak san iritasyon, gratèl oswa woujè.\*

\*Kontakte enfimiyè WOC/NSWOC ou si w remake iritasyon, gratèl oswa woujè

## Prepare po a Baryè ak Étui

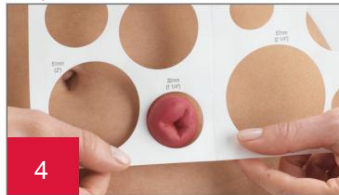

Mezire stoma a lè l sèvi avèk yon gid pou mezire stoma anvan chak aplikasyon baryè (si w ap itilize yon ouvèti baryè ki gen menm gwosè, ale nan etap 7).



Trase gwosè baryè detèmine ouvèti a sou baryè po a. Baryè po a ta dwe anfòm kote po a ak stoma rankontre. Verifye ke pa gen okenn po ki montre ant baryè po a ak stoma a pou ede anpeche flit ak iritasyon po.



Sèvi ak ti sizo koube, ak anpil atansyon koupe yon ouvèti nan baryè po a matche ak modèl la. Pa koupe pi lwen pase liy lan sou revètman lage a.



Retire revètman lage nan baryè po a.

## Ajoute kole oswa Bag si sa nesèsè



Si sa apwopriye, aplike yon bag baryè oswa yon pèl mens nan keratin alantou ouvèti a swiv enstriksyon yo pou itilize.

## Aplike Nouvo a Pouching sistèm



Santre baryè po a ouvèti sou stoma a nan yon fòm dyaman.

Pou baryè tep entoure, retire papye a fè bak sou tou de bò epi peze adezif sou po a.




Aplike presyon dou sou baryè po a pou apeprè yon minit pou pi bon adezyon.



Tache plastwon an nan baryè po a lè w angaje kwen pi ba yo nan bride sach la ak bride baryè po a.

Peze de bride yo ansanm. Yon seri "klik" pral konfime ou ap fè li kòrèkteman. Pa gen plis "klik" vle di sak la an sekirite.

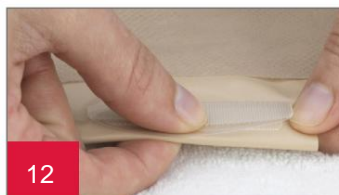

Fèmen anba sak. Si w ap itilize yon sak drenaj, fèmen **fèmen microseal Lock 'n Roll** nan pati anba a oswa aplike yon kranpon pou sak. Al gade nan Konsèy pou Swen Fèmti Microseal Lock 'n Roll la pou plis enfòmasyon sou fason pou itilize sistèm fèmen sa a.



Si w ap itilize yon sak urostomy, asire w ke tiyo drenaj la fèmen.

Teardrop la sou tiyo drenaj la ta dwe fè fas a bò kò a nan sak la.

Si ou vle, aplike bouchon ki bay nan bwat la.

KONSÈY SWEN STOMI

## De-moso Pouching Sistèm

KONSÈY	LIS PWODWI	STOCK NO
<ul style="list-style-type: none"><li>Po alantou stoma ou a pa ta dwe montre okenn siy gratèl oswa maleng</li><li>Yon pwofesyonèl swen sante ta dwe evalye po irite</li><li>Si w ap itilize yon bag baryè oswa yon keratin, aplike li sou bò kolan nan baryè po a akote ouvèti stoma a jan yo di l.</li><li>Pwobableman stoma w ap anfle apre operasyon an. Li ka pran plizyè semèn oswa mwa pou stoma a retresi nan gwoès pèmanan li</li><li>Stoma a ka yon ti kras senyen lè w netwaye li</li><li>Itilize nan ti sèvyèt pwoteksyon po yo pa rekòmande anba baryè po mete pwolonje paske yo ka diminye tan mete</li></ul>	<p><b>De-moso Pouching Sistèm</b></p> <p>Baryè po _____</p> <p>Étui _____</p> <p><b>Pwodwi pou Telefòn</b></p> <p>Bag baryè _____</p> <p>Senti ostomi _____</p> <p>Kole _____</p> <p>Lubrifyan deyodoran _____</p> <p>Stoma poud _____</p> <p>Lòt _____</p>	

### Enstriksyon adisyonèl

---

---

---

---

---

---

---

---

Pou kesyon pwodwi, bezwen echantiyon, oswa kesyon klinik detaye konsènan pwodwi nou yo nan peyi Etazini, rele **1.888.808.7456**. Nan Kanada rele **1.800.263.7400**.

Po a ta dwe pwòp epi sèk anvan ou aplike baryè po a.

Anvan w itilize, asire w ke w li Enstriksyon pou Itilizasyon yo pou jwenn enfòmasyon konsènan Itilizasyon Prewwa, Kontr, Avètisman, Prekosyon, ak Enstriksyon.

Logo Hollister, Lock 'n Roll ak "Healthy po. Rezilta pozitif." se mak komèsyal Hollister Incorporated. © 2018 Hollister Incorporated. 923081-1118 US-00666

**Hollister Incorporated**  
2000 Hollister Drive  
Libertyville, Ilinwa 60048  
1.800.323.4060

[www.hollister.com](http://www.hollister.com)

**Hollister limite**  
95 Mary Street  
Aurora, Ontario L4G 1G3  
1.800.263.7400

