

Adapt Medical Adhesive Spray

General Information

- Increases the adhesion of skin barriers and/or tape
- Helps to provide a secure seal to minimize leakage and maintain skin health
- Medical grade silicone adhesive

Using Medical Adhesive Spray

1. Prepare your pouching system as instructed.
2. Remove the protective release liner from the adhesive (skin barrier and/or tape) on your pouching system.
3. Spray directly on to the adhesive side of the pouching system. The 360° spray technology minimizes waste, and will allow you to spray from any angle.
4. Allow to dry for 5 minutes, and apply your pouching system as instructed.

WARNING: Use only as directed. Do not apply this product directly to the skin or wounds.



Adapt Medical Adhesive Spray
Stock No 7730



Spray directly on to the adhesive, and allow to dry for 5 minutes. Then apply pouch as instructed.

FREQUENTLY ASKED QUESTIONS

<p>Is it necessary to use the Adapt medical adhesive spray on the entire skin barrier and tape?</p>	<p>You can spray it on the entire adhesive surface of the pouching system. However, if there are particular areas that are not adhering well, you can use it only in those areas.</p>
<p>What is the best way to spray the adhesive?</p>	<p>Hold can so that the arrow faces the product to be sprayed. Spray in short bursts until the surface of the product is coated evenly.</p>
<p>Can a second coat be applied?</p>	<p>Yes, if necessary. Remember to wait 5 minutes for the second coat to dry before applying your pouching system.</p>
<p>Can I spray it directly on my skin?</p>	<p>This product should not be sprayed directly on skin and it is not necessary in order for it to be effective.</p>
<p>Will the adhesive be difficult to remove from the skin?</p>	<p>The medical adhesive should come off easily when the pouching system is removed. If needed, the Adapt medical adhesive remover spray or Adapt universal remover wipe can help with ease of removal.</p>

Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe

General Information

- Helps remove adhesive skin barriers and/or tape
- Removes adhesive residue from the skin
- Skin Friendly Formulation
 - Silicone-based
 - Alcohol free
 - No sting

Using Adhesive Remover Spray

1. Peel back one edge of the skin barrier or tape border.
2. Hold the can so that the arrow faces the base of the skin barrier or tape, and spray until the adhesive releases. The 360° spray technology minimizes waste, and will allow you to spray from any angle.
3. Ensure your skin is dry and apply your new ostomy pouching system as usual.

Using Universal Remover Wipes

1. Tear open the package and take out the wipe.
2. Press or rub the wipe against your skin as you peel off your skin barrier and/or tape.
3. Press or rub the wipe over any areas of the skin where tape residue remains.
4. Ensure your skin is dry and apply your new ostomy pouching system as usual.



Adapt Medical Adhesive Remover Spray/Adapt Universal Remover Wipe

Stock No 7737/7760



Spray - Spray until the skin barrier releases.



Wipes - Rub the wipe over skin as you peel off skin barrier.



Wipes - Clean off remaining tape residue.

FREQUENTLY ASKED QUESTIONS

Is an adhesive remover always needed?	No, this product is optional. Use it to remove adhesive residue from your skin, or to make it easier to remove your skin barrier or tape.
Will this be uncomfortable to use?	This silicone-based product is formulated to be a no-sting alternative. When used correctly, the adhesive remover will help to make the removal of your pouching system easier and help reduce the potential for skin stripping.
Do I need to wash the adhesive remover off of my skin?	The adhesive remover evaporates quickly and dries without residue, so the next barrier can attach securely. If you wash your skin after use be sure to rinse and dry thoroughly before the next pouch application.

WARNING: Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US, call **1.888.808.7456**. In Canada call **1.800.263.7400**.

Routine follow-up with your healthcare professional is recommended.

Prior to use, be sure to read the Instructions for Use for information regarding Intended Use, Contraindications, Warnings, Precautions, and Instructions.

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2019 Hollister Incorporated. 923073-0619 US-00218

Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048
1.800.323.4060

www.hollister.com

Hollister Limited
95 Mary Street
Aurora, Ontario L4G 1G3
1.800.263.7400