

FastCast OLS

A V.I.P. Treatment Solution
for Your DFU Patients*



V Vascular Management
with low-level
compression sock



I Infection Management and Prevention
with the included Hydrofera Blue Ready foam
and Endoform dermal template dressings



P Pressure Relief
with FastCast OLS
total contact cast system




FastCast OLS
total contact cast system



* Each healthcare provider should apply the FastCast OLS system based on individual patient needs.

FastCast OLS system provides statistically

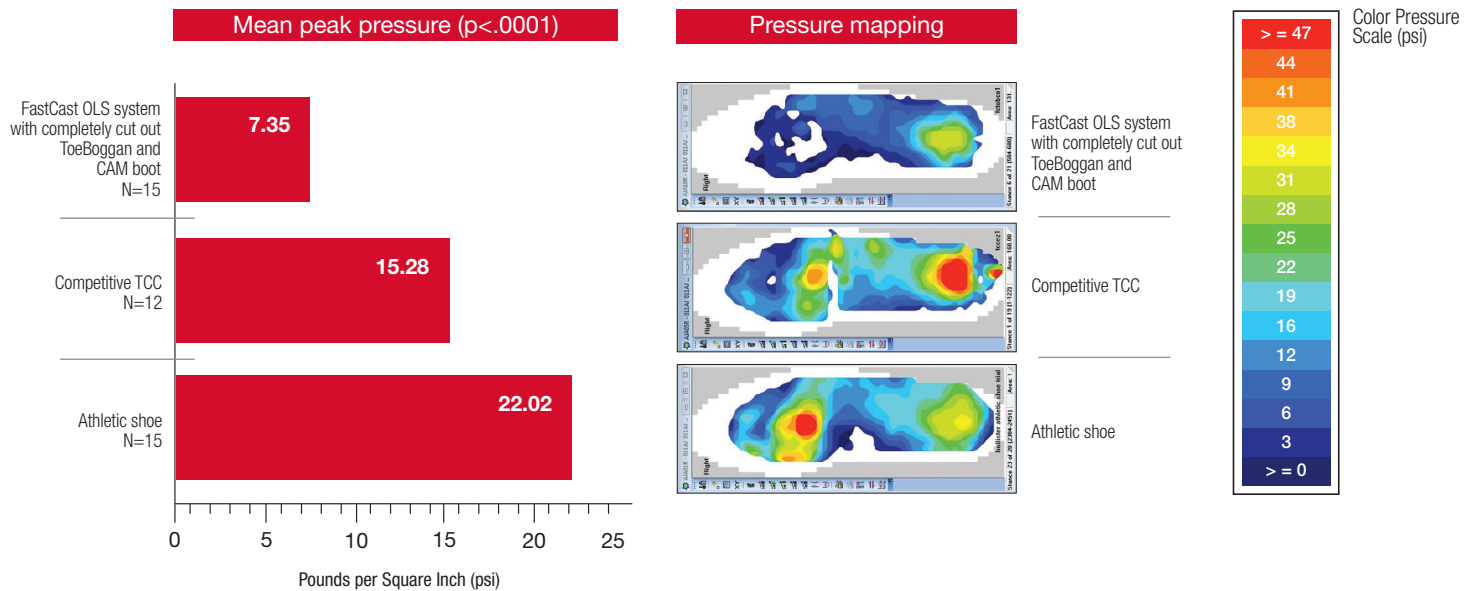

- ▲ Compared to a competitive TCC, FastCast OLS system provides significant pressure reduction.
- Methodology: both were applied according to the manufacturer's instructions for use* by healthcare providers experienced in total contact casting.




Managing V.I.P.s helps improve DFU wound healing outcomes²

Off-Loading Consensus Guidelines² in November 2014 state that vascular management (V), infection management and prevention (I), and pressure relief (P) are essential to DFU healing.²

- Evidence suggests that if V.I.P.s are aggressively managed, then the wound-healing trajectory will progress.²
- Without proper pressure relief, successful healing will be unlikely, even when using advanced therapeutics.²


*CAM boots were modified at the discretion of the principal investigator for purposes of the study.

significant pressure reduction¹

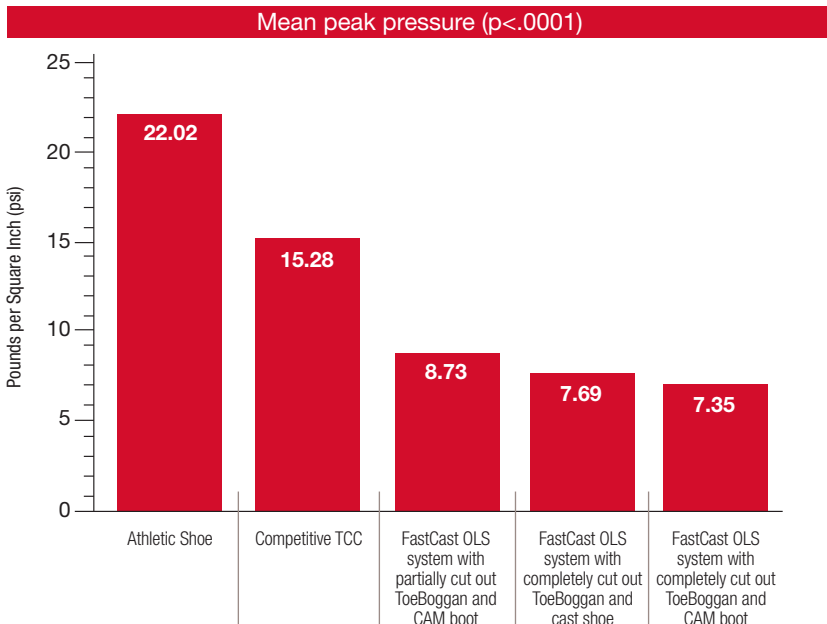

- ▲ If the ToeBoggan footplate is partially or completely cut out, the FastCast OLS system provides significantly greater pressure reduction than the competitive TCC.
- Significant difference in pressure reduction was observed between the athletic shoe (control) and the three FastCast OLS system application techniques shown in the chart above.
- Using a cast shoe showed comparable pressure reduction compared to a CAM boot.

NOTE: The following data is provided for informational purposes only. Please follow IFU for optimal results.

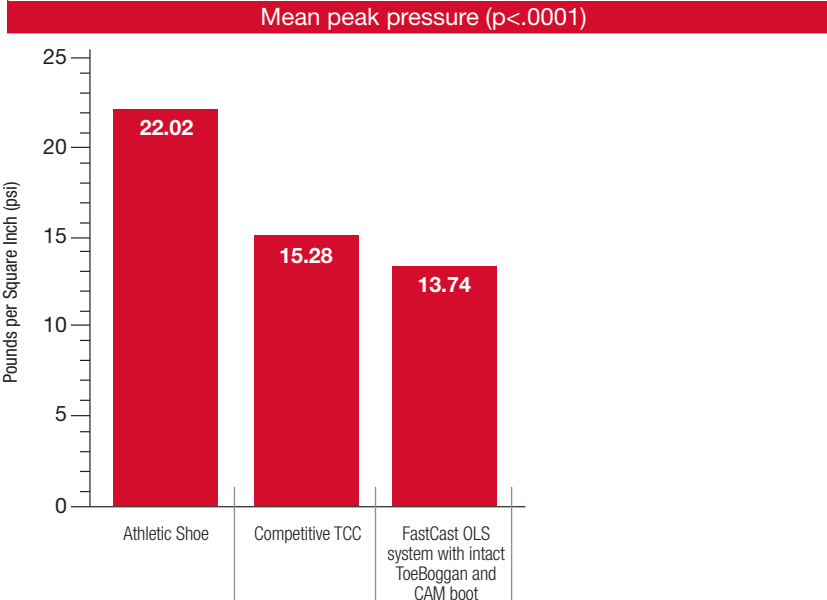

- ▲ Even when the ToeBoggan footplate is left intact, the FastCast OLS system provides comparable pressure reduction to the competitive TCC.



The Versatile and Durable DFU Management System

- No wait to ambulate – patient can immediately walk once CAM boot is in place
- Use of TCC promotes patient adherence¹
- The convenience kit includes award-winning dressings:*
 - Endoform dermal template provides native extracellular matrix and matrix metalloproteinase reduction³ to aid in wound closure
 - Hydrofera Blue Ready foam provides broad-spectrum antibacterial activity and exudate management⁴

Outpatient Wound Care Center**

- CPT 29445 (Application of Total Contact Cast) – Physician Reimbursement⁵
- APC 5102 (links with CPT 29445) – Medicare Clinic Reimbursement⁶

Physician Office**

- CPT 29445 (Application of Total Contact Cast) – Physician Reimbursement⁵
- Q4038 (Fiberglass Cast Supplies) – Supply Reimbursement⁷

Indications for Use

FastCast OLS system is indicated for the following wound types:

- Diabetic, plantar or neuropathic foot ulcers
- Post-operative care
(i.e., Charcot reconstruction, delayed primary closures)
- Charcot neuroarthropathy
- Pre-ulcerative conditions

Caution: Prior to use of the FastCast OLS total contact cast system, be sure to read the product Instructions for Use package insert that accompanies the product.

Rx Caution: Federal (USA) law restricts this device for sale or on the order of a physician or licensed healthcare professional. Refer to Instruction for Use for contraindications, warnings, precautions and possible complications.

References:

1. Data on file. Compared to TCC-EZ[®].
2. Snyder, RJ, et al. The Management of Diabetic Foot Ulcers through Optimal Off-Loading Building Consensus Guidelines and Practical Recommendations to Improve Outcomes; *Journal of American Podiatric Medical Association*. 2014; 104(6): 555 – 567.
3. Negron L, Lun S, May BC. Ovine forestomach matrix biomaterial is a broad spectrum inhibitor of matrix metalloproteinases and neutrophil elastase. *Int Wound J*. 2012 Nov 1.
4. Data on file. Hydrofera Blue antibacterial foam Instructions for Use.
5. OPTUM[™] EncoderPro.com Expert. https://www.encoderpro.com/epro/medicarePhysFeeSchedHandler.do?_k=101*29445&_a=listRelated Outpatient facility payment rates based on 2015 National Medicare Average Payments. *Current Procedure Terminology [CPT] copyright 2015 American Medical Association. All Rights Reserved. CPT[®] is a registered trademark of the AMA. Physician CPT[®] rates reflect a conversion factor of \$35.7547 [CR9081].
6. <https://www.cms.gov/Medicare/Medicare-Fee-for-Service-Payment/HospitalOutpatientPPS/Hospital-Outpatient-Regulations-and-Notices-Items/CMS-1633-FC.html?DLPage=1&DLEntries=10&DLSort=2&DLSortDir=descending>.
7. Durable Medical Equipment, Prosthetics, Orthotics, and Supplies (DMEPOS), Revised January 2015 Fee Schedule. <http://www.cms.gov/Medicare/Medicare-Fee-for-Service-Payment/DMEPOSFeeSched/DMEPOS-Fee-Schedule-Items/DME15-A.html?DLPage=1&DLSort=2&DLSortDir=descending>.

FastCast OLS total contact cast system ordering info:

Stock No.	Product Description
FC0001	FastCast OLS 50" - 1 Cast Kit
FC0001-8	FastCast OLS 50" - 8 Cast Kits
FC0002	FastCast OLS 50" - 5 Cast Kits & 1 Medium Boot
FC0003	FastCast OLS 50" - 5 Cast Kits & 1 Large Boot
FC0006	FastCast OLS 56" - 1 Cast Kit
FC0006-8	FastCast OLS 56" - 8 Cast Kits
FC0007	FastCast OLS 56" - 5 Cast Kits & 1 Large Boot
FC0008	FastCast OLS 56" - 5 Cast Kits & 1 Extra Large Boot
FC0011	FastCast OLS Small CAM Boot
FC0012	FastCast OLS Medium CAM Boot
FC0013	FastCast OLS Large CAM Boot
FC0014	FastCast OLS Extra Large CAM Boot
FC0024	FastCast OLS 4" Cast Roll, Replacement

*Endoform dermal template 2013 innovation awards *Podiatry Today* and *OWM*, and Hydrofera Blue Ready foam 2014 innovation awards *Podiatry Today* and *OWM*.

**The reimbursement information is intended to provide general information concerning coding of Hollister products only. Hollister does not guarantee coverage or payment for any TCC products. The ultimate responsibility for proper coding, satisfying reimbursement requirements, and obtaining reimbursement remains with the provider. Coding and coverage policies and guidelines are complex, can vary from one carrier or region to another, and are updated frequently. Providers should check with their local carriers or intermediaries often and should consult with counsel, or a reimbursement specialist for coding, coverage, or billing questions.



Hollister Incorporated
2000 Hollister Drive
Libertyville, Illinois 60048 USA
1.888.740.8999

www.hollister.com

The Hollister Logo, FastCast OLS and ToeBoggan are trademarks of Hollister Incorporated. Hydrofera Blue Ready is a trademark of Hydrofera LLC. Endoform is a trademark of Mesynthes Limited. All other trademarks and copyrights are the property of their respective owners.
©2016 Hollister Incorporated.
922780-0216

FastCast OLS

total contact cast system