
INSTRUCTIONS/使用说明/GEBRAUCHSANLEITUNG / BRUGSANVISNING / INSTRUCCIONES / KÄYTTÖOHJEET / MODE D'EMPLOI / ISTRUZIONI / 사용법
GEBRUIKSAANWIJZING / BRUKSANVISNING / BRUKSANVISNING / คำแนะนำ / 説明

US	Male Urinary Pouch
GB	男用尿袋
CN	男用尿袋
DE	Externer Urinableiter für Männer
DK	Selvløbende urinpose til mænd
ES	Bolsa Urinaria Masculina
F	Miesten Virtsankeräyssussi
FR	Collecteur Urinaire Masculin
IT	Sacca Urinaria per uomo

KR	남성용 소변 파우치
NL	Urineopvangzakke voor mannen
NO	Urinoppsamlingspose for menn
SE	Urinpåse för män
TH	ถุงเก็บปัสสาวะสำหรับผู้ชาย
TW	男用尿袋

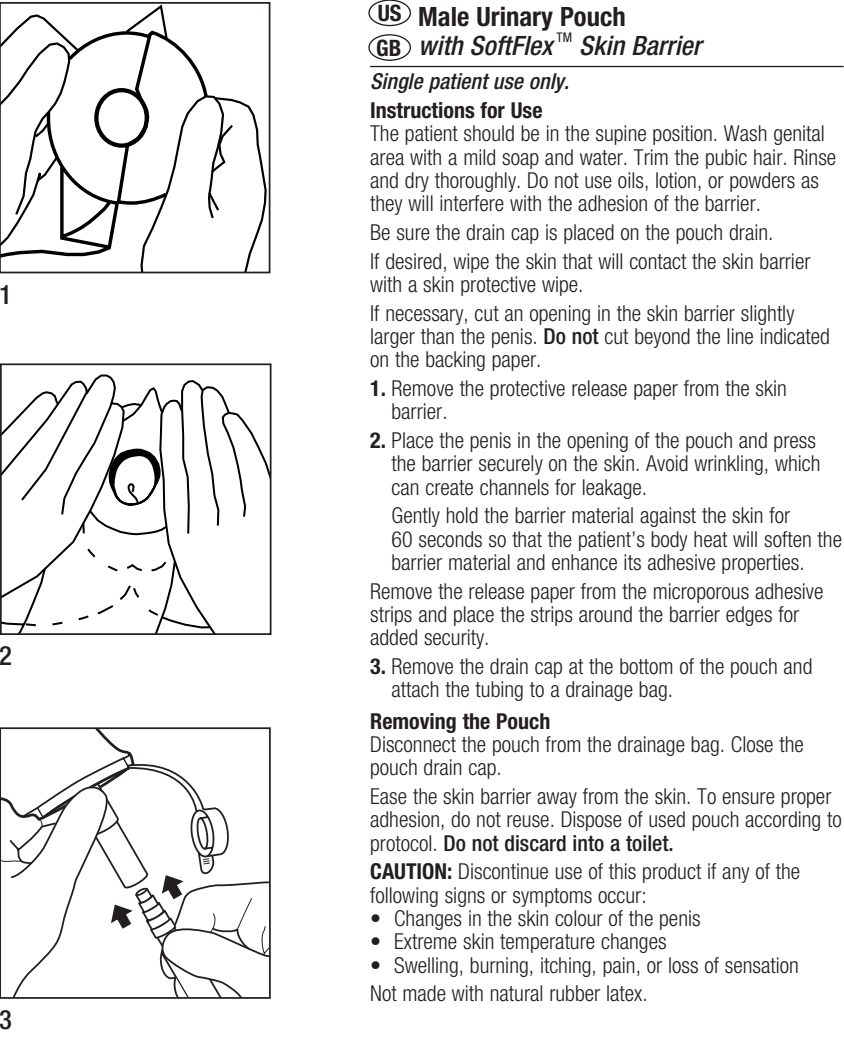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Adapt™ Skin Protective Wipe Instructions

Alcohol-free – No Sting!

Helps prevent irritation associated with removal of adhesives. Acts as a protective interface between skin and bodily wastes and fluids when used with adhesive products.

Precautions: For external use only.

- To be used on intact skin only.** Do not apply to severely irritated or eroded skin, open cuts, or sores.
- Avoid contact with clothing or other textiles while the liquid dries.
- Avoid contact with eyes.

If skin irritation occurs, cease use immediately, and seek medical advice. Users with any known sensitivities to any of the ingredients should not use this product.

Directions: Skin should be clean and dry prior to application of the Adapt skin protective wipe. Apply a thin uniform coating. Let dry. Protective coating forms, which bonds to skin. Coating does not have to be removed from skin when changing appliance. If the Adapt skin protective wipe is used in an area with skin folds or other skin-to-skin contact, make sure the skin contact areas are separated to allow the coating to thoroughly dry before returning to normal position.

INGREDIENTS: WATER, PVP, BUTYLENE GLYCOL, VPVA COPOLYMER, ALOE VERA DECOLORIZED WHOLE LEAF EXTRACT, SODIUM HYALURONATE, OCTOXINOL-9, PEG-12 DIMETHICONE, DISODIUM EDTA, DIAZOLIDINYL UREA, IODOPROPENYL BUTYLCARBAMATE.

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