
Hollister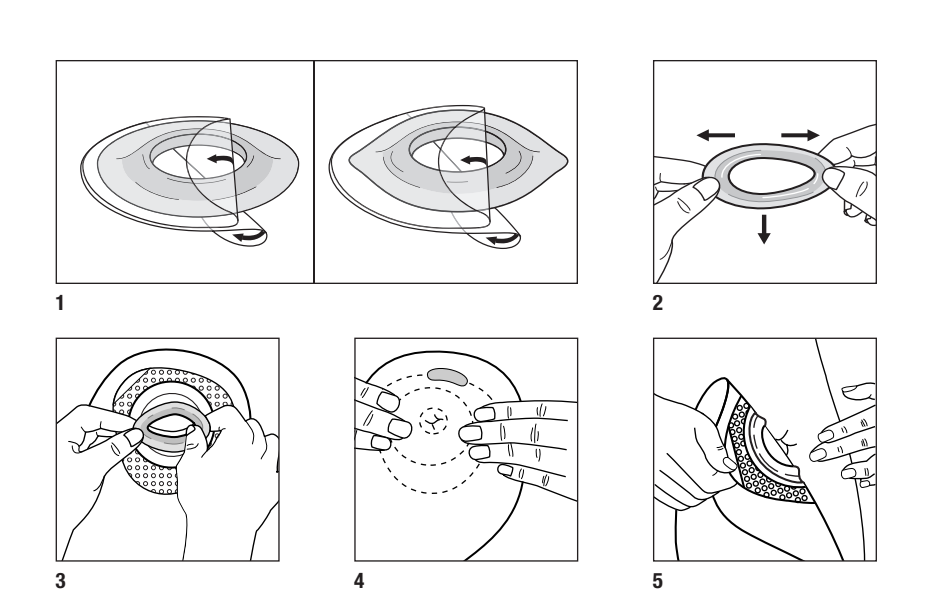

🇺🇸 🇬🇧 Instructions for Adapt™ Convex Barrier Rings

Adapt convex barrier rings can be used under skin barriers, adhesives and in combination with paste.

Directions for Application

The skin should be clean and dry before application.

- Both barrier ring protective coverings can be removed at the same time or, if desired, the bottom covering can be removed (flat side of ring), retaining the top covering.

Note: if wearing gloves and both protective coverings are removed, be sure the gloves are moist.

- If required stretch the Adapt convex barrier ring before applying to the skin barrier – up to 5 mm in either direction.
- Press the Adapt convex barrier ring to the skin barrier to ensure adhesion. If the top protective covering has been retained, remove it.
- Center (Centre) the skin barrier with Adapt convex barrier ring around the stoma. Press the skin barrier with Adapt convex barrier ring down around the stoma and against the skin. Hold in place for several seconds.

Removal

- Gently peel away from skin, working downwards while pressing lightly against the surrounding skin.

Note: If convexity is not needed all around the stoma the ring can be cut.

INGREDIENTS: PRODUCT MADE FROM HYDROCOLLOID ADHESIVE AND POLYMER/CO-POLYMER PLASTICS.

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©2018 Hollister Incorporated.