US GB Medical Adhesive Remover – Instructions for Use

Silicone-based, alcohol free – No Sting! Spray from any direction.

- 1.Spray remover around the edge of the adhesive.
- 2.After a few seconds gently peel the adhesive away from the skin using additional Adapt™ medical adhesive remover if necessary.

Dries in seconds. Once dried, Adapt medical adhesive remover will not affect adhesion of products applied later.

For ostomy products, peel back one edge of skin barrier or tape/adhesive border, aim the spray away from the skin and towards the base of the barrier or adhesive surface.

For maxillo-facial prosthetic devices, saturate a gauze pad and wipe the adhesive away. Remover evaporates quickly, additional applications to gauze may be required.

CAUTION: Pressurized (pressurised) container. Protect can from sunlight and do not expose to or store at temperatures exceeding 120°F (50°C). Do not puncture or incinerate container, even after use. DO NOT USE IN CONFINED SPACES.

WARNING: Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.

FLAMMABLE: Do not spray on a flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed.

CONTENTS: HEXAMETHYLDISILOXANE WITH NITROGEN

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