US GB Medical Adhesive – Instructions for Use

For ostomy appliances.

Hold adhesive can so that the arrow points to the product to be sprayed. Spray adhesive on device from any direction. Spray in short bursts until the surface of the product is coated evenly with adhesive. Wait about five minutes for the adhesive to dry before applying the skin barrier or device to the skin. If necessary, apply a second coat.

WARNING: Use only as directed. Do not apply this product directly to the skin or wounds.

FLAMMABLE: Do not spray on a flame or any incandescent material, electrical appliance or source of heat. Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed.

CAUTION: Pressurized (pressurised) container. Protect can from sunlight and do not expose to or store at temperatures exceeding 120°F (50°C). Do not puncture or incinerate container, even after use.

CONTENTS: MEDICAL GRADE ADHESIVE POLYMER POLYDIMETHYLSILOXANE IN HEXAMETHYLDISILOXANE SOLVENT WITH NITROGEN.

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©2018 Hollister Incorporated.