en 2 Piece Urostomy Pouch

da 2-dels urostomipose

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fr-ca Sac d'urostomie 2 pièces

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©2019 Hollister Incorporated.
912612-1019

en INSTRUCTIONS FOR USE

- WARNINGS**
Reuse by reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.
- Notes:** Excess mucus from a urostomy stoma may cause the pouch to drain slower than expected. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Product Description: Part of a two-piece ostomy pouching system. Designed to be used with a corresponding skin barrier with the same flange size. The pouch is designed to be emptied. The pouch is most suitable for urostomies. It has a tap to empty liquid. Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is part of a two-piece ostomy pouching system intended to collect output from a stoma (e.g., urostomy) or skin opening (e.g., wound, fistula).


MD Medical Device
(as applicable)

O Flange (coupling ring)


Not made with natural rubber latex.

CE mark as required


To attach pouch to barrier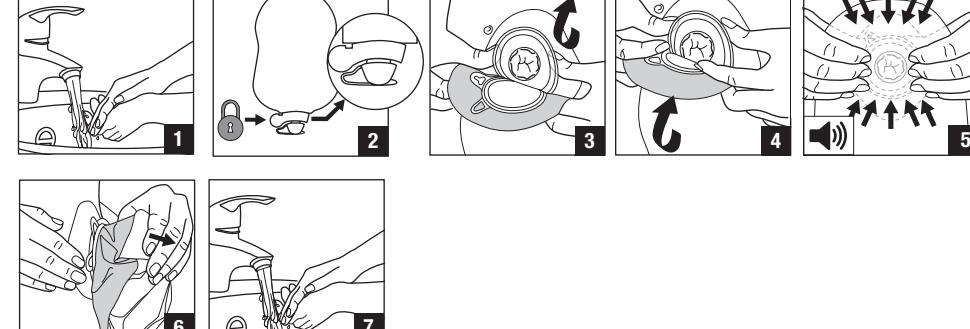


To drain and close pouch

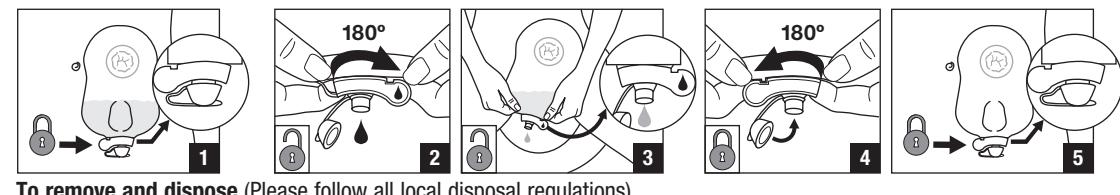To remove and dispose (Please follow all local disposal regulations)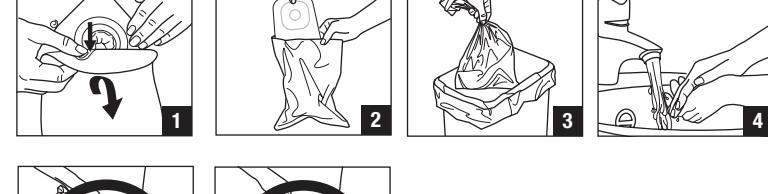

To use the Urostomy drain tube adapter (optional)

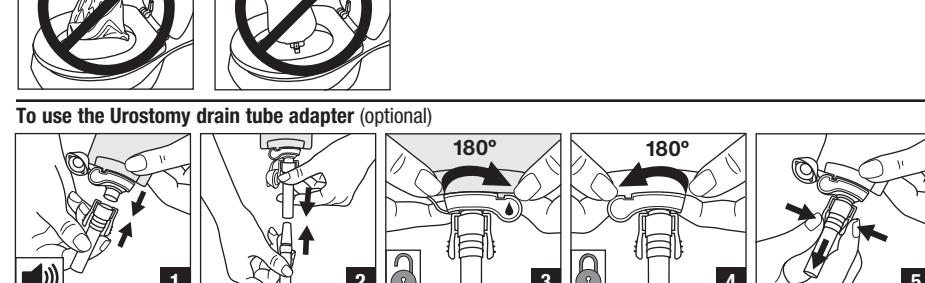Single patient - multiple use. Rinse with clean water and leave to dry. Discard when visibly damaged.

Sharing the urostomy drain tube adapter with multiple users creates risk of infection.

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