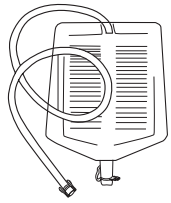
REF 5551

en High Output Collection Bag

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www.hollister.com/symbolglossary

912717

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

EC REP

©2020 Hollister Incorporated.
912717-920

en INSTRUCTIONS FOR USE

Contents

- High output collection bag with tubing and attached connector.
- Hanger

Product Description: High output collection bag that connects directly to a Hollister high output drainable ostomy pouch without the need for additional connectors. Features punch holes for attachment to a nightstand or bed hanger. Designed to be emptied. Hanger fixation to support a collection or night drainage bag. Most suitable for colostomies or ileostomies including high output stomas or post-operative use.

Product is made of polymer/copolymer plastics, and nonwoven fabric.

Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to collect output from Hollister high output drainable ostomy pouches. The hanger is an accessory intended to secure a collection or night drainage bag to a bed frame or a stand.

WARNINGS

Disinfection and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Precaution

Single patient – multiple use. Empty then rinse bag and tubing with clean water, and hang to dry. Wipe outside of bag and tubing with damp cloth as needed. Discard when visibly damaged or stained. Sharing this product with multiple users creates risk of infection.

Notes: Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

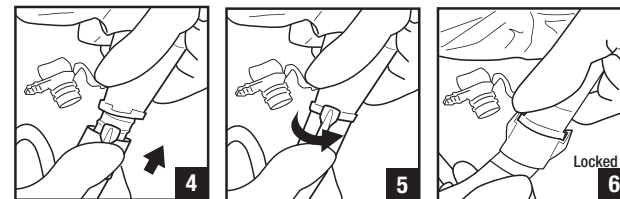
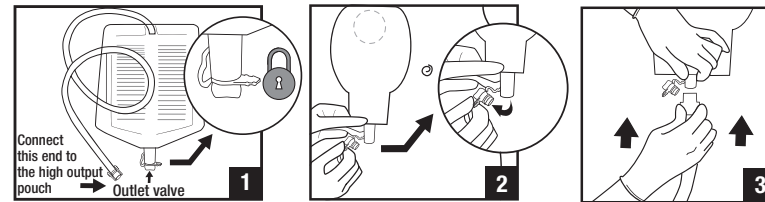
Single patient – multiple use.

Medical Device (as applicable).


Not made with natural rubber latex.
CE mark as required

To connect to a high output drainable ostomy pouch


Applying the hanger fixation

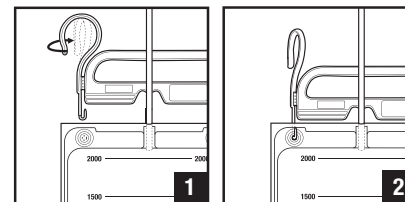Please keep the bag off the floor. Discard when visibly damaged or stained.


To drain and close

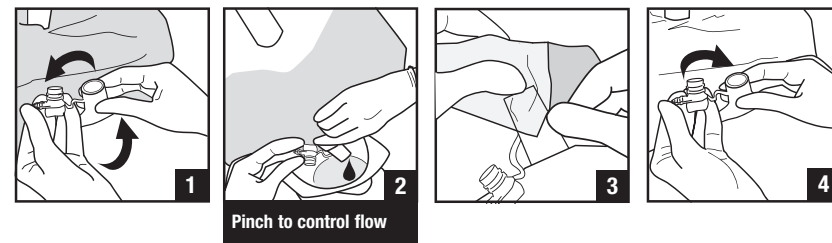To remove and dispose (Please follow all local disposal regulations)

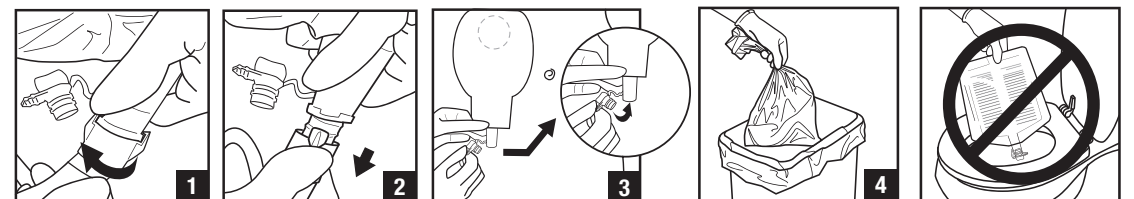Remove hanger fixation

For reference only. Unauthorized use, disclosure or copying is prohibited.

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