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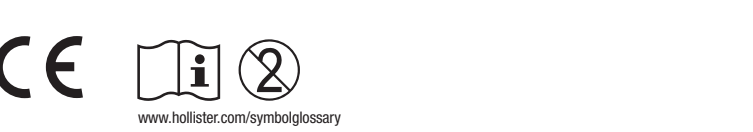
www.hollister.com/symbolglossary

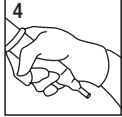
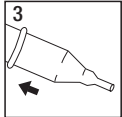
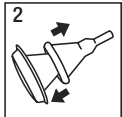
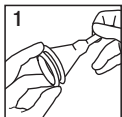
en INSTRUCTIONS FOR USE
Contents Male Incontinence Sheath (Catheter), Self-Adhesive
Product Description The self-adhering male external sheath (catheter) is designed for the management of adult male urinary incontinence.
Made of silicone and adhesive.
Indications and Intended Use The product is an accessory intended to channel urine from the penis to extension tubing and urinary collection bags for adult males with urinary incontinence.
WARNINGS Reuse of a single use product may create potential risk to the user such as poor adhesion, leakage, and skin irritation.
Precautions Do not use on irritated or compromised skin.
Do not use if allergic reaction occurs.
Ensure proper fit as instructed. If the sheath (catheter) size is incorrect or leakage occurs, skin damage may occur (e.g., irritation, broken skin, rash, or redness). This may decrease wear time.
Follow directions to remove if experiencing swelling, numbness, discomfort, pain, discoloration, or abnormal appearance.
Notes For good hygiene, change sheath (catheter) daily.
To help ensure proper adhesion, apply to clean, dry intact skin. Do not use lotions, powders, or ointments on the skin as these products may interfere with adhesion of the product. Contact a healthcare practitioner for help with the product or with skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

To prepare
1 Verify correct product size prior to use. Trim pubic hair if necessary.
2 Wash penis with mild soap and warm water. Dry thoroughly.

To apply
1 Open package at perforation. Remove sheath (catheter) from plastic insert, if present.
2 Place rolled end over the end of the penis, leaving a small space between the end of the penis and the cone of the sheath (catheter). If applying to an uncircumcised penis, allow foreskin to remain in its natural position. Take care not to pinch or retract the foreskin.
3 Slowly unroll onto the penis with as little wrinkling as possible until product is fully extended.
4 Gently squeeze to properly seal adhesive to the skin. Do not attempt to reposition the product. If possible, avoid leaving a rolled "collar" around the base of the penis.

To connect this product to a drainage system, refer to the instructions for use packed with that product.

To remove and dispose
Disconnect from the drainage system. Gently roll the sheath (catheter) forward and off the penis. If necessary, apply a warm wet compress (such as a wet washcloth) around the sheath (catheter) to help loosen the adhesive.
Please follow all local disposal regulations.

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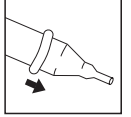
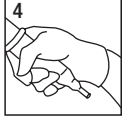
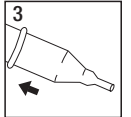
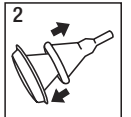
For information on disposal, please refer to the instructions for use packed with the product.

For information on disposal, please refer to the instructions for use packed with the product.

For information on disposal, please refer to the instructions for use packed with the product.

For information on disposal, please refer to the instructions for use packed with the product.

For information on disposal, please refer to the instructions for use packed with the product.

For information on disposal, please refer to the instructions for use packed with the product.

CE mark as required
MD Medical Device (as applicable)

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