en	Adapt Ostomy Convex Barrier Rings
da	Adapt konvekse stomitætningsringe
de	Adapt Konvekse Stoma-Hautschutzringe
es	Anillos de barrera convexos para ostomía Adapt
fi	Adapt – kuperat ihonsuojarenkaat avanteelle
fr	Anneaux protecteurs convexes Adapt pour stomie
fr-CA	Anneaux convexes de champ protecteur de stomie Adapt
it	Anelli barriera convessi per stomia Adapt
nl	Adapt convexe pastaringen voor stoma's
no	Adapt konvekse tetningsringer for stomi
sv	Adapt konvexa tätningsringar för stomi

www.hollister.com/symbolglossary

Hollister®

912855

Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

de Adapt Konvekse Stoma-Hautschutzringe

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Medizinprodukt
(sofern anwendbar).

en Adapt Ostomy Convex Barrier Rings

Contents: Convex barrier rings
Product Description: Barrier rings designed to improve the fit of an ostomy pouching system. For use when convexity is needed. Made with hydrocolloid skin adhesive.
Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to help prevent drainage from getting under the skin barrier. Barrier rings help ostomy products conform to the skin.
WARNINGS: Reuse of a single use product may create potential risk to the user such as poor adhesion. Reprocessing, cleaning, disinfection, and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Notes: The product may be cut, stretched, or stacked together to help improve fit of a pouching system. Ensure the barrier ring opening is appropriate for your stoma size. If convexity is not needed all around the stoma the ring can be cut. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

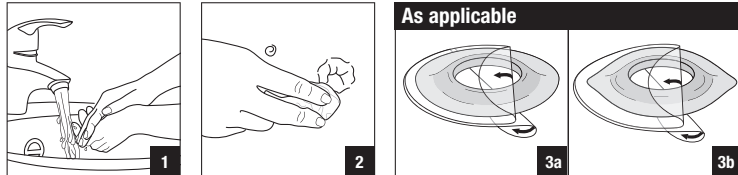
Not made with natural rubber latex.
CE mark as required


Medical Device
(as applicable).

INSTRUCTIONS FOR USE

To prepare skin and product

To apply

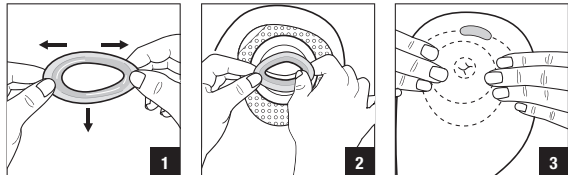To remove and dispose (Please follow all local disposal regulations)

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©2020 Hollister Incorporated.