en Urinary Leg bags

es Bolsas de orina de pierna

it Sacche urinarie da gamba

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en INSTRUCTIONS FOR USE

Contents

- Urinary leg bag with drain valve and attached tubing
- Urinary leg bag straps

Product Description: Urinary leg bag designed to be emptied or to connect to a night drainage bag. Straps to hold a urinary leg bag securely to the thigh or calf.

Not made with natural rubber latex. Product is made of polyvinylchloride (PVC), low-density polyethylene (LDPE), polyester, and elastic polyurethane fabric.

Indications and Intended Use: A urinary leg bag collects and contains urine. The straps secure a urinary leg bag to the leg. These accessory products are for individuals with urinary incontinence or a urinary stoma.

WARNINGS

- Disinfection and sterilisation (sterilization) may compromise the product. This may result in odour (odor) or leakage.

Precautions

- Single patient – multiple use. Sharing this product with multiple users creates a risk of infection.

Between uses, empty and rinse leg bag with lukewarm water or clean per healthcare provider recommendations. Hang to dry away from heat. Wipe outside of bag and tubing with damp cloth as needed. Clean the connector with an alcohol wipe before each use. Discard when visibly damaged or stained.

Notes: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

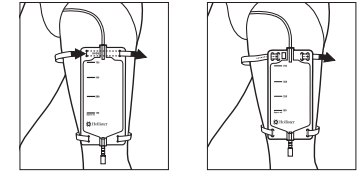
MD Medical Device (as applicable).
CE mark as required

Applying the leg bag

Ensure the drain tap is closed. Remove cap from stepped mount connector if present. Connect the stepped mount connector to the sheath (catheter). Make sure tubing is not kinked or obstructed.

Applying the leg bag straps

Adjust the straps so they are comfortable and secure.

Precautions

- To ensure adequate circulation, do not overtighten.
- Check the area under the straps regularly for skin irritation or trauma.
- To prevent improper drainage do not attach leg bag to knee.

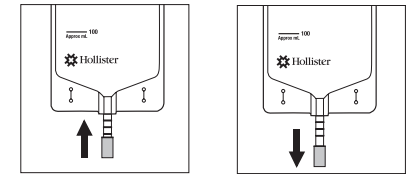
Discard when visibly damaged or stained.

Draining the leg bag

Tap closed

Tap open

To remove and dispose (Please follow all local disposal regulations)
Remove leg bag straps.

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©2020 Hollister Incorporated.