en Adhesive Remover Wipes

Contents: Adhesive Remover Wipes
Product Description: Silicone-based remover wipe. Alcohol free – no sting.
Indications and Intended Use: This product is an accessory to an ostomy pouching system intended to ease removal of tapes, barriers, cements and their adhesive residue from the skin.
INGREDIENTS: Hexamethyldisiloxane
WARNINGS To help reduce the potential for infection and/or other complications, do not reuse.
EXTREMELY FLAMMABLE: Keep away from sources of ignition – No Smoking. Keep out of the reach of children. Use only as directed.
Precautions For external use only. Use only as directed. Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, and vagina). Discontinue use if irritation occurs.
Notes:
Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.
INSTRUCTIONS FOR USE
Skin should be clean and dry prior to application. Open packet and immediately begin using the wipe. Starting at the edge, gently peel back the skin barrier or adhesive as you apply the Adapt universal remover at the junction between the adhesive and the skin. Continue to wipe and peel the barrier or adhesive back until it has been completely removed. When the barrier or tape is completely removed from the skin, gently wipe off any remaining residue.
Please follow all local disposal regulations.
Store in a cool, dry place.

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