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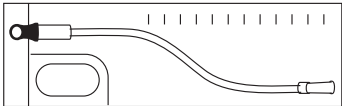
Pre-Lubricated Urethral Intermittent
Catheter with Integrated Collection Bag

Advance Plus Pocket™

Manufactured in India by:
Hollister Medical India Private Limited
Neutral Code # HR-676-B(H)


Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland

 **Hollister**

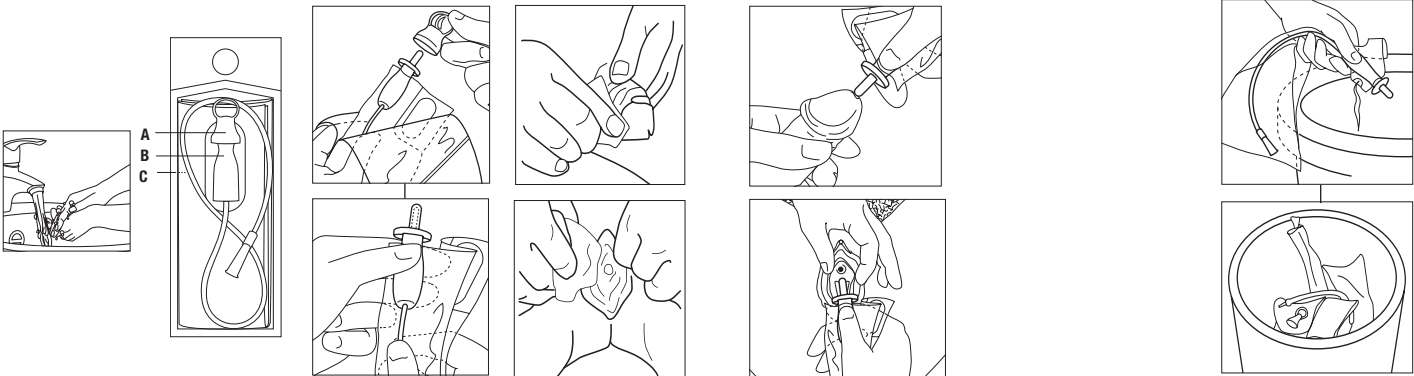©2022 Hollister Incorporated.
913077-122

For reference only. Unauthorized use, disclosure or copying is prohibited.

en Instructions

To prepare
Assemble all equipment and prepare the area. Wash your hands as advised by your healthcare practitioner. Position yourself to carry out intermittent catheterisation. Manipulate your clothing to access your genitalia.

Fully fold back the packaging by peeling the clear side of the package. Remove the protective cap from the protective tip (A). Hold the gel reservoir (B) in one hand and with the other hand move the catheter forward until tip of the catheter fills the protective tip, taking care that the catheter does not protrude from the tip. Lay the catheter on the opened package. The protective tip should not be outside the opened package.

Male
Hold the penis with your non-dominant hand and don't let go until you have finished the procedure. Pull back foreskin (if present) and cleanse the glans and opening of urethra as instructed by your healthcare practitioner.

Female
Spread the labia with two fingers of your non-dominant hand. Cleanse around opening of urethra as instructed by your healthcare practitioner.

With your dominant hand hold the gel reservoir gently.
Male
Using your forefinger and thumb of your non-dominant hand squeeze penis gently to open the urethral opening. With the penis stretched and in upright position, insert the protective tip until the base comes in contact with the urethral opening. Slightly press the base onto the glans to help ensure the protective tip does not slip out of the urethra. Relax foreskin (if present).
Female
Part the labia with two fingers of your non-dominant hand. With your dominant hand, insert the protective tip until the base comes in contact with the urethral opening. Keep the protective tip in place. Release the labia.

It will be helpful to occasionally straighten the collection bag while advancing the catheter. This will prevent the catheter from backing out of the urethra. Continue to insert the catheter forward until the urine starts to flow. Once urine flow has stopped, slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely.

To empty the urine, tear the bag at the notch (C). Empty the contents into a toilet or suitable receptacle.

To dispose
Catheter and integrated collection bag may be disposed of in a waste bin. DO NOT FLUSH DOWN TOILET. Wash your hands.

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en Pre-lubricated urethral intermittent catheter with integrated collection bag

Contents

Pre-lubricated urethral intermittent catheter with integrated collection bag

Product Description:

- Ready-to-use, pre-lubricated with gel, single use intermittent catheter
- Protective tip and integrated urine collection bag support no touch insertion technique
- Gel reservoir
- 2 smooth catheter eyelets
- Colour-coded funnel

	Catheter effective length: the length of the catheter that can be inserted into the urethra.		
40 cm/ 16" catheter	39 cm	390 mm	15.4"

Not made with natural rubber latex. This product consists of: polyethylene/EVA collection bag, thermoplastic elastomer tip, water soluble lubricating gel, and tubing not made with DEHP.

Indications and Intended Use: This intermittent catheter is a flexible tubular device that is inserted through the urethra by male, female and paediatric patients who need to drain urine from the bladder.

WARNINGS

Do not use if package is damaged or opened. To help reduce the potential for infection and/or other complications, do not reuse. If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

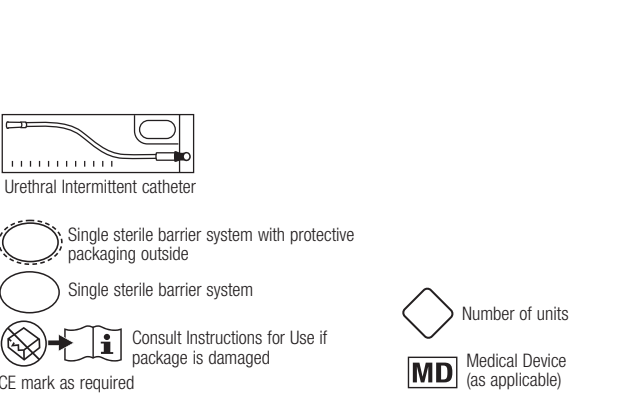
Precautions: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra
- Prostatitis – inflammation of the prostate gland
- Epididymitis – inflammation of the epididymis (testicle tube)

Self-catheterisation should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterisation frequency varies by person, the recommended frequency of your catheterisation should be provided by your healthcare practitioner. For any other questions about your catheterisation, please contact your healthcare practitioner.

NOTES: Contact a healthcare practitioner for help with continence care products. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

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