CeraPlus™

30

REF

79402

Hollister®
©2023 Hollister Incorporated
913247-223

R90-45E

EC REP

Hollister ULC
Foxford Road, Ballina
Co. Mayo, Ireland
Manufactured for:
Hollister Incorporated
2000 Hollister Drive
Libertyville, IL 60048 USA

www.hollister.com/symbolglossary

en Barrier Extenders

Instructions for use

Contents

Hydrocolloid skin barrier extenders

Product Description

CeraPlus barrier extenders are infused with ceramide to protect the skin. Barrier extenders are thin, flexible strips designed to help an ostomy pouching system conform to uneven skin folds and contours. They are designed to help reduce skin barrier edge lifting and provide an extra sense of security. Product is made with hydrocolloid skin adhesive suitable for general wear.

Indications and Intended Use

This product is an accessory to an ostomy pouching system intended to help an ostomy product conform to the skin.

WARNINGS

Reuse of a single use product may create potential risk to the user such as poor adhesion.

Reprocessing, cleaning, disinfection, and sterilisation may compromise the product. This may result in odour or leakage.

Notes:

To help ensure proper adhesion, apply to clean, dry intact skin. Do not use oils or lotions. These products may interfere with adhesion of the skin barrier. Contact a stoma care nurse or healthcare practitioner for help with the ostomy pouching system or skin irritation. In case of serious injury (incident) in relation to your use of the product, please contact your local distributor or manufacturer, and your local competent authority. For more information, see www.hollister.com/authority or contact EC Rep or local distributor.

Medical Device
(as applicable)

Number
of Units

Not made with natural rubber latex.
CE mark as required

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
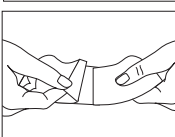
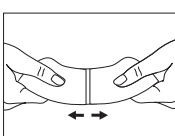
en To position

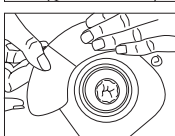da Placering

en To remove and dispose (Please follow all local disposal regulations)

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